


**Huvitz**

---

**SMART BLOCKER**  
**HBK-410**

---

USER MANUAL

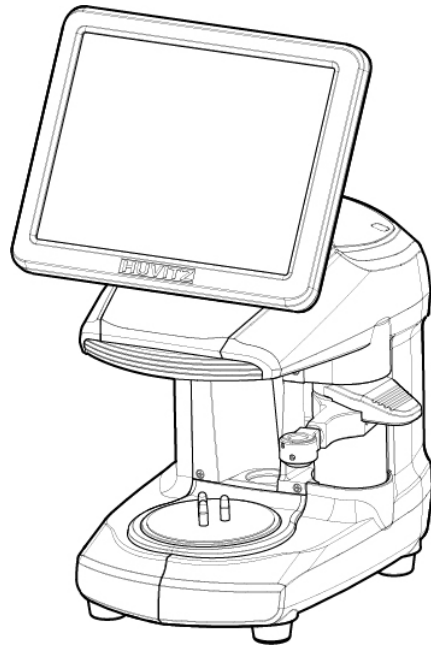

## ■ IMPORTANT NOTICE

---

This product may malfunction due to electromagnetic waves caused by portable personal telephones, transceivers, radio-controlled toys, etc. Be sure to avoid having the above objects, which affect the normal operation of the product, brought near the product.

The information in this publication has been carefully checked and is believed to be entirely accurate at the time of publication. HUVITZ assumes no responsibility, however, for possible errors or omissions, or for any consequences resulting from the use of the information contained herein.

HUVITZ reserves the right to make changes in its products or product specifications at any time and without prior notice, and is not required to update this documentation to reflect such changes.

**9000ENG0071-A**  
**Ver 1.0 (Mar. 2019)**

©2019 HUVITZ Co., Ltd.

38, Burim-ro 170beon-gil, Dongan-gu, Anyang-si, Gyeonggi-do, 14055, Republic of Korea

All rights reserved.

Under copyright laws, this manual may not be copied, in whole or in part, without the prior written consent of HUVITZ Co., Ltd.

# I CONTENT

<b>1. Introduction.....</b>	<b>6</b>
1.1. Main Features .....	6
1.2. System Configuration .....	7
1.3. Classification.....	7
<b>2. Safety Information.....</b>	<b>8</b>
2.1. Introduction .....	8
2.2. Safety Symbols.....	9
2.3. Environmental Considerations .....	10
2.4. Safety Precautions .....	12
<b>3. System Overview .....</b>	<b>15</b>
3.1. Front View.....	15
3.2. Rear View .....	15
3.3. Standard Accessory.....	16
<b>4. Installation.....</b>	<b>17</b>
4.1. Installation Procedure.....	17
4.2. How to connect the device .....	17
<b>5. Smart Blocker User Interface.....</b>	<b>19</b>
5.1. Main Screen.....	19
5.1.1. Side Button.....	19
5.1.2. Job Management .....	22
5.1.3. Frame Information .....	24
5.1.4. Layout Option.....	24
5.1.5. Edging Option .....	28
5.1.6. Lens Information .....	33
5.1.7. Function Button.....	37
5.2. Blocking Screen.....	38
5.2.1. Layout Information .....	39
5.2.2. Lens Type.....	39
5.2.3. Tool Button .....	40
5.2.4. Sending Job .....	40
5.2.5. Exit .....	40
5.3. Digital Scan.....	42
5.4. Job Manager .....	45
5.4.1. Job selection tab.....	46
5.4.2. Function Button.....	46
5.4.3. Waiting Job .....	47
5.4.4. Working Job .....	47
5.4.5. Completed Job.....	48
5.4.6. Job Search.....	49

5.5.	Digital Pattern .....	51
5.5.1.	Enlargement/Reduction .....	52
5.5.2.	Near Vision Area Enlargement/Reduction .....	53
5.5.3.	Rotation .....	54
5.5.4.	Partial Modification .....	56
5.5.5.	Toggle R/L Sync.....	58
5.5.6.	Reset.....	59
5.5.7.	Exit .....	59
5.6.	Hole Editor .....	60
5.6.1.	Upper Side of Hole Editor .....	61
5.6.2.	Working Area.....	61
5.6.3.	Preview .....	63
5.6.4.	Function Buttons .....	64
5.6.5.	List of Hole/Slot.....	71
5.6.6.	Property of Hole/Slot .....	74
5.7.	Job Editor.....	75
5.7.1.	Layout Option.....	75
5.7.2.	Edging option .....	76
5.8.	Step Bevel Editor.....	77
5.8.1.	Step Bevel Processing Mode .....	78
5.8.2.	Partial Step Bevel Editing Area .....	78
5.8.3.	Partial Step Bevel Option .....	81
5.8.4.	Full Step Bevel Editing Area.....	82
5.8.5.	Full Step Bevel Option .....	84
5.8.6.	Fine Tuning .....	84
5.8.7.	Preview .....	88
5.8.8.	Current Position Information.....	89
5.8.9.	Safety beveling & Polishing option.....	90
5.8.10.	Reset & Apply Button.....	91
5.9.	Bevel Height Editor.....	92
5.9.1.	Editing Area.....	93
5.9.2.	Preview .....	95
5.9.3.	Current Position Information.....	96
5.9.4.	Edging Option .....	97
5.9.5.	Reset & Apply Button.....	97
<b>6.</b>	<b>Configuration .....</b>	<b>98</b>
6.1.	Preference .....	99
<b>7.</b>	<b>System Maintenance.....</b>	<b>106</b>
7.1.	Brightness Test & Calibration .....	106
7.2.	Save Image.....	107
7.3.	Test Mode .....	108
7.4.	Date & Time.....	109
7.5.	Touch Test.....	110
7.6.	Configuration Management .....	111
7.7.	SW Upgrade .....	112
7.8.	Maintenance Code .....	113

7.9. Maintenance Tips ..... 114

    7.9.1. Fuse Replacement..... 114

    7.9.2. Regular maintenance ..... 114

    7.9.3. Cleaning ..... 114

**8. Troubleshooting Guide .....115**

**9. Specifications and Accessories.....116**

    9.1. Standard Accessories ..... 116

    9.2. Specifications..... 116

    9.3. Drawings of System..... 117

**10. SERVICE INFORMATION .....118**

# 1

---

## Introduction


This product is the device for blocking, which means mounting the adapter on the lens. The operator adjusts the position of the lens using camera image and information for edging and performs the blocking manually.

### 1.1. Main Features

- This product provides a different guide for blocking, depending on the type of lens such as a lens marked by lens-meter, bifocal lens and progressive lens. And the guide can be adjusted. This makes blocking more convenient.
- 9.7" LCD display and touch screen are harmonized to offer the actual size (Frame, PD and FPD etc.) display and the maximum convenience for everyday use.
- User can design all information about layout and edging at one page, and send it to other two edger machines at a same time.
- This device provides the Digital Pattern function to transform the frame shape and the Hole Editor function to support drilling.
- The Digital Scanning technology can scan a frame shape for a demo lens without a conventional tracer. This provides you an enormous convenience and a time saving.
- Frame data is saved in SD memory card permanently. Therefore, you can load them whenever you want.

## 1.2. System Configuration

- Interface the Smart Blocker with frame reader and edger.

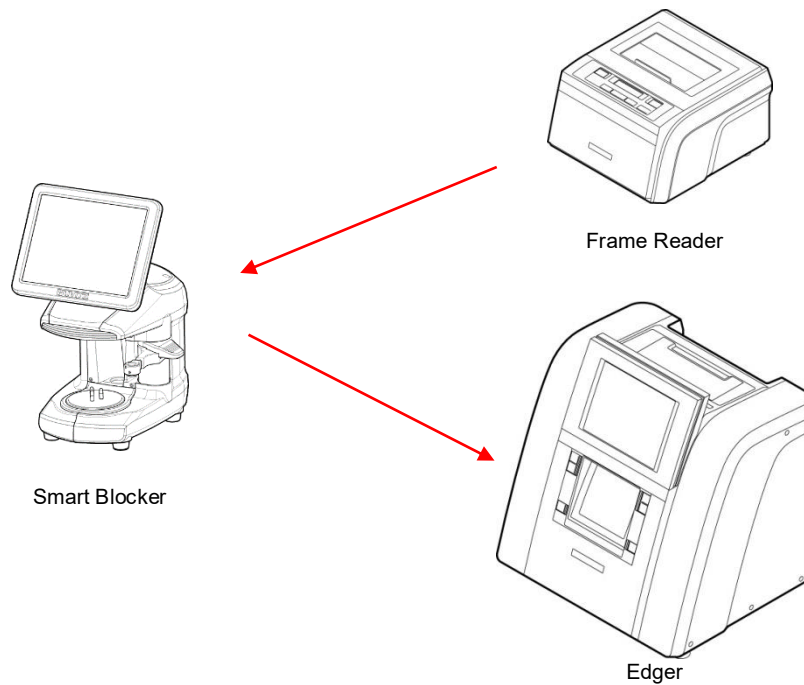

## 1.3. Classification

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

- Class A equipment  
Class A equipment is intended for use in an industrial environment. In the documentation for the user, a statement shall be included drawing attention to the fact that there may be potential difficulties in ensuring electromagnetic compatibility in other environments, due to conducted as well as radiated disturbances.
- Protection against electric shock: Class I (when earthed)
- Measurement Category: CAT II
- Pollution Degree: 2

# 2

---

## Safety Information


### 2.1. Introduction

Safety is everyone's responsibility. The safe use of this machine is largely dependent upon the installers, users, operators, and managers. It is prerequisite to read and understand these specifications before installing, using, cleaning, fixing or revising. Fully understanding the whole instructions must be the first priority. For this reason, the following safety notices have been placed appropriately within the text of this manual to highlight safety related information or information requiring special emphasis. All users, operators, and maintainers must be familiar with and pay particular attention to all signs of Warnings and Cautions.



---

"Warning" indicates the presence of a hazard that could result in severe personal injury, death or substantial property damage if ignored.

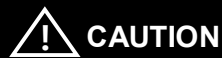

---

"Caution" indicates the presence of a hazard that could result in minor injury, or property damaged if ignored.



---

This is used to emphasize essential information.  
Be sure to read this information to avoid operating the device incorrectly.

## 2.2. Safety Symbols

The International Electrotechnical Commission (IEC) has established a set of symbols, which are listed below. This applies only to the instrument that has the certification symbol printed explicitly on the product label or sticker.



I and O on power switch represent ON and OFF respectively.  
(I et O sur l'interrupteur d'alimentation représentent ON et OFF respectivement.)



This symbol identifies caution, risk of danger. Ensure you understand the function of this control before using it. Control function is described in the appropriate User's or Service Manual.  
(Ce symbole identifie la prudence, le risque de danger. Assurez-vous de bien comprendre le fonctionnement de ce contrôle avant de l'utiliser. La fonction de contrôle est décrite dans le manuel de l'utilisateur ou de service approprié.)



Protective Earth Connection  
(Connexion de protection à la terre)

### Disposal of your old appliance



1. When this crossed-out wheeled bin symbol is attached to a product it means the product is covered by the European Directive 2002/96/EC.  
(Lorsque ce symbole de poubelle à roulettes barrée est attaché à un produit, cela signifie que le produit est couvert par la directive européenne 2002/96 / CE.)
2. All electrical and electronic products should be disposed of separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.  
(Tous les produits électriques et électroniques doivent être éliminés séparément du flux de déchets municipaux via des centres de collecte désignés par le gouvernement ou les autorités locales.)
3. The correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health.  
(La mise au rebut correcte de votre ancien appareil aidera à éviter les conséquences négatives potentielles sur l'environnement et la santé humaine.)
4. For more detailed information about disposal of your old appliance, please contact your city office, waste disposal service or the shop where you purchased the product.  
(Pour plus d'informations sur l'élimination de votre ancien

---

appareil, veuillez contacter votre mairie, le service d'élimination des déchets ou le magasin où vous avez acheté le produit.)

---



Alternating Current  
(Courant alternative)

---

### 2.3. Environmental Considerations

Avoid the following environments for operation or storage:



Where the machine is exposed to water vapor.  
Don't operate the machine with wet hands Indoor use only.

---



Where the machine is exposed to direct sunlight.

---




A place where the equipment can be exposed to direct ultraviolet.

---



Where there are big changes in temperature.

---

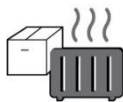


Where there is a hot equipment nearby.

---



Where the humidity is extremely high or there is a ventilation problem.

---

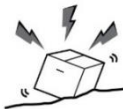

Where the machine is exposed to excessive shocks or vibrations.

---




Where the machine is exposed to chemical material or explosive gas.

---



Be cautious so that things like dust and metal do not fall inside the machine.

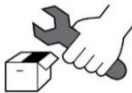


Don't disassemble or open the product. HUVITZ does not take responsibility for the possible problems



Be careful not to block the fan of the machine.



Don't plug the AC power cord into the outlet unless all parts of the machine are completely connected. Otherwise, it will cause severe damage on the machine.

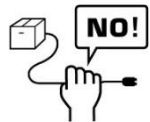

Pull out the power cord with holding the plug, not the cord.  
To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

---

This instrument must be followed by these following conditions:

- Operation
  - An ambient temperature range of 10 °C ~ 40 °C
  - A relative humidity range of 30% ~ 90%
  - An atmospheric pressure range of 800 ~ 1060hpa
  - An indoor use, Altitude up to 2000 m
- Transportation
  - An ambient temperature range of -40 °C ~ 70 °C
  - A relative humidity range of 10% ~ 95% (with non-condensing)
  - An atmospheric pressure range of 500 ~ 1060hpa
- Storage
  - An ambient temperature range of -10 °C ~ 55 °C
  - A relative humidity range of 10% ~ 95% (with non-condensing)
  - An atmospheric pressure range of 700 ~ 1060hpa
- Temporary Overvoltage
  - Short-term temporary overvoltage: 1440V
  - Long-term temporary overvoltage: 490V

Please avoid where the equipment is exposed to excessive shocks or vibration.

## 2.4. Safety Precautions

This machine has been developed and tested according to safety standards as well as national and international standards. This guarantees a very high degree of safety for this device. HUVITZ is legally required to inform the users of all the information regarding safety. Observance of the instructions is the requirement for the safety. Therefore, please read carefully all instructions before switching on this machine. For more detailed information, please contact our Customer Service Department or one of our local agencies.

1. This equipment must not be used (a) in an area that is in danger of explosions and (b) in the presence of flammable, explosive, or volatile solvent such as alcohol, benzene or similar chemicals.
2. Do not place or store this machine in humid area. Do not expose the device to water splashes, dripping water, or sprayed water. Do not place containers with fluids, liquids, or gases on top of this machine.
3. The machine must be operated by a trained and qualified person or under his or her supervision.
4. Repair of this machine must be conducted by HUVITZ's service technicians or other authorized persons.
5. Maintenance by users must observe the User's Manual and Service Manual. Any additional maintenance may only be performed by HUVITZ's service technicians or other authorized persons.
6. Manufacturers are responsible for the safety, reliability, and performance of this machine only when the following requirements are fulfilled: (1) When the machine has been installed in a proper area, following the manual. (2) When the machine has been operated and maintained according to the manual and service manual.
7. Manufacturers are not responsible for the damages caused by unauthorized alterations. Such tampering will forfeit any rights to receive services during the term of guarantee.
8. This machine must be connected with the accessories supplied by HUVITZ. If you are to use other accessories, their safety or usability must be checked and proved by their manufacturers or HUVITZ.
9. Only those who have undergone proper training and instructions are authorized to install, use, operate, and maintain this machine.
10. Keep the User's Manual and Service Manual in a place easily accessible at all times for persons operating and maintaining the equipment.
11. Do not apply excessive force to cable connections. If the cable does not connect easily, make sure

that the connector (plug) is appropriate for the receptacle (socket). If you caused any damage to a cable connector(s) or receptacle(s), let the damage(s) be repaired by an authorized service technician.

12. Please do not pull on any cable. Always grab the plug when disconnecting cables.
13. Before you use, check the exterior of the machine and its conditions.
14. Do not block any ventilation outlet necessary for proper heat dissipation.
15. If smoke, sparks or any abnormal noise or smell is noticed coming from the machine, please switch the power off immediately and pull out the plug.
16. Dispose of used batteries according to the regulations. If not, there is the possibility of an explosion.
17. To avoid the risk of electric shock, this machine must only be connected to protective earth.
18. Do not place the machine where it is difficult to operate the disconnecting device. (disconnecting device: power cord)
19. The machine may be impaired if it is used in a manner not specified by the manufacturers or manual.

**WARNING**

- Do not put your hands into the machine while blocking. Otherwise, it may cause serious injury.
- Do not block without lens adaptor, otherwise the lens may be broken or scratched.
- Do not disassemble the product without proper training. It may cause electric shock, serious injury during the operation or cause malfunction of the product.
- The product should be properly installed and maintained at where the ground is even and flat. Otherwise, it may affect the normal operation of the product.

## CAUTION

---

- When moving the machine, first fix the stage and check whether the power supply is off. Then, should lift the bottom of the product with both hands.
- When setting down the machine, make sure not to be interfered with the obstacles. Set down the product slowly in order to prevent any injury of human body or damage to the product.
- When moving the machine, make sure the locking device and the screw for the cover are properly tightened.
- When wrapping the machine, use recommended packaging and shock-absorbing materials in order to prevent damage during the transportation.
- Be sure to use the standard accessories or tools provided together with the product for the maintenance. Otherwise, it may cause the malfunction of the product.
- Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

## NOTE

---

This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules.


These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.


# 3

## System Overview


### 3.1. Front View

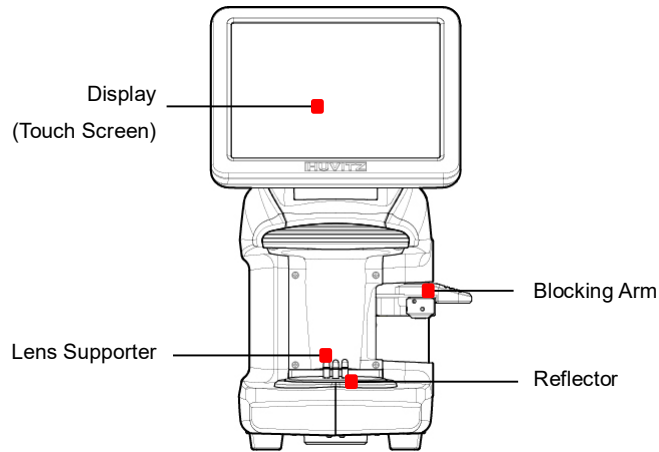


### 3.2. Rear View

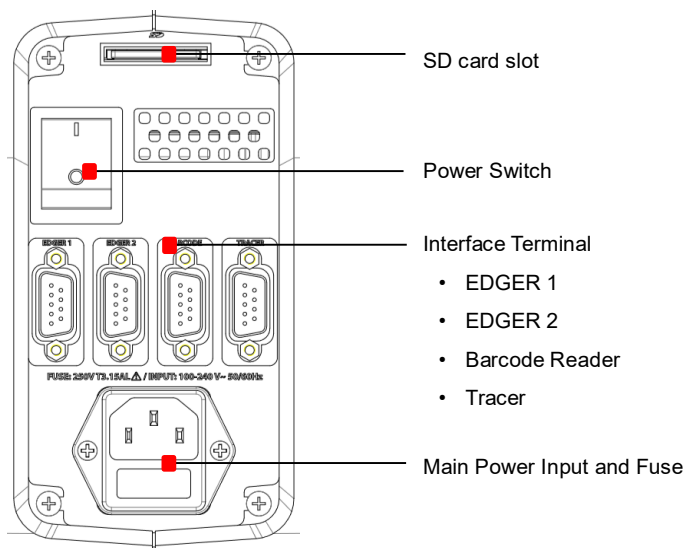


### 3.3. Standard Accessory

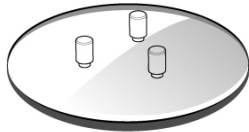

Lens Supporter

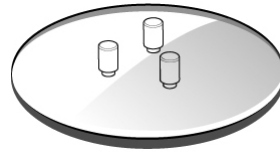

Lens Supporter for Frame Change and Digital Scanning

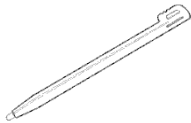

Touch Pen



#### NOTE

- Please use genuine lens adaptor tape (LEAP-III). Otherwise, the lens axis may move.
- We do not provide a sticker for Super Hydrophobic Lens, but you should use qualified ones, or the lens axis may move.

# 4


## Installation

### 4.1. Installation Procedure

- ① Remove the shock-absorbing materials from the packing box and take out the smart blocker carefully.
- ② Remove the shock-absorbing materials from the blocking arm.
- ③ Plug the Power Cable into the socket at the rear side of the product and turn on the power.
- ④ If the initial indicator disappears and the main screen is shown on the LCD, check if the touch screen is responding correctly by touching icons or else.

### 4.2. How to connect the device

In this manual, HFR-8000X frame reader and HPE-8000X edger are used as an example.

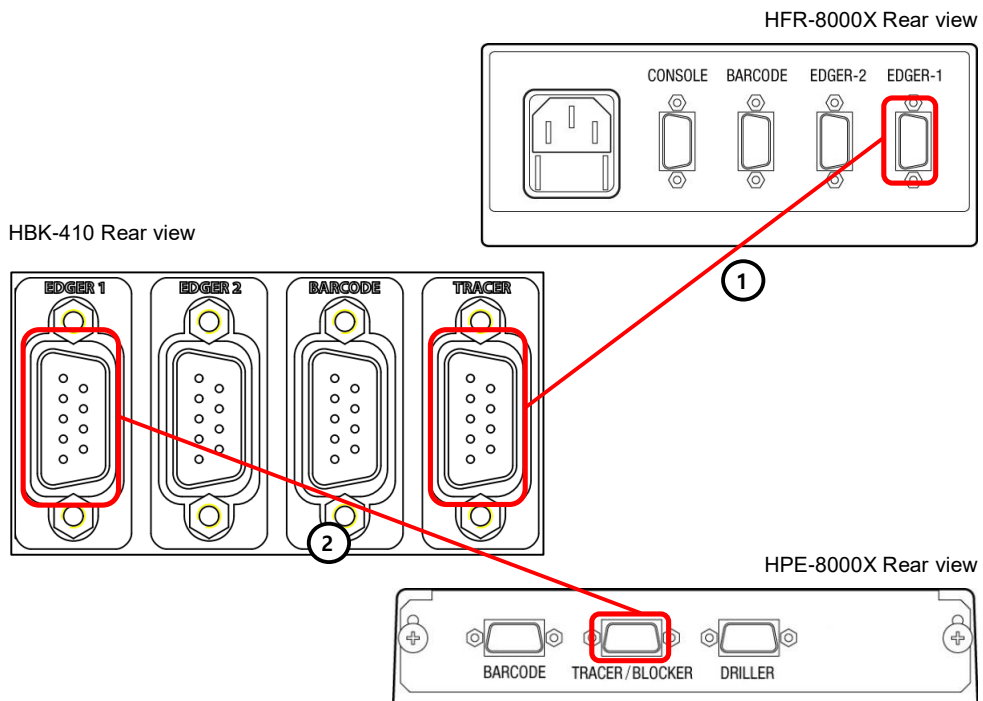

- ① Connect a 9-pin serial cable to the EDGER 1 port of the HFR-8000X and the TRACER port of the HBK-410.
- ② Connect a 9-pin serial cable to the EDGER 1 port of the HBK-410 and the TRACER/BLOCKER port of the HPE-8000X.
- ③ After connecting the cables, turn on the power of all the devices and check the initialization process is performed properly.
- ④ Read the frame from the frame reader and transfer the data to the blocker. And then check the blocker is receiving the job data normally.
- ⑤ From the blocker, open the job data and switch to the blocking screen and then send the job data to the edger. (For the UI screen description, refer to 'Chapter 5.2 Blocking Screen')
- ⑥ Make sure that the edger receive the job data from the blocker normally.



## NOTE

---

Keep the shock-absorbing material in the box and use them again when you move the machine.


# 5

## Smart Blocker User Interface


### 5.1. Main Screen



#### 5.1.1. Side Button

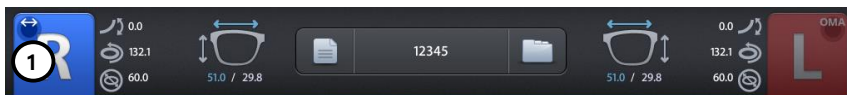

① Press the R button to select right side.

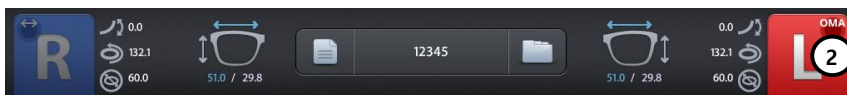


② Press the L button to select left side.

**NOTE**

- Each side of frame shape is exchanged by pressing the R button for a while.




- Left and Right sides are exchanged. The other job options remain unchanged.

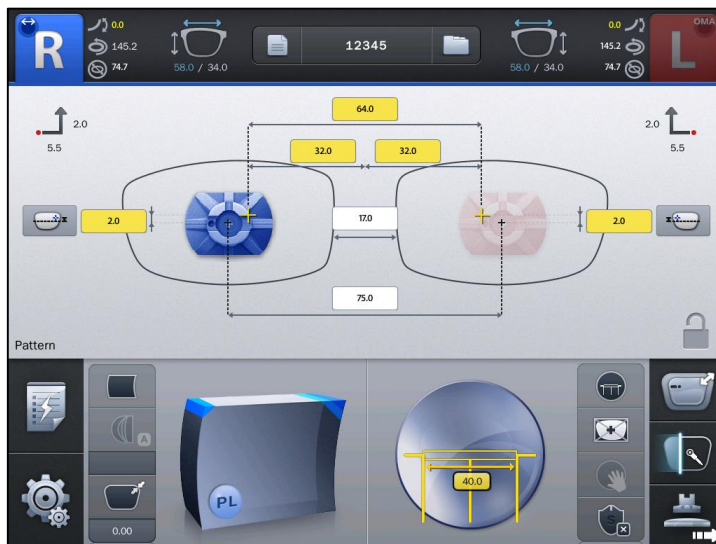

- The function of exchange R/L is useful when you selected wrong side while reading demo lens or patterns.
- In addition to R/L exchange, R/L shape mirroring function is available by setting the preference option. (Refer to 'Chapter 6.1 Preference')


 **NOTE**

- OMA compatible job data file can be imported by pressing the L button for a while.

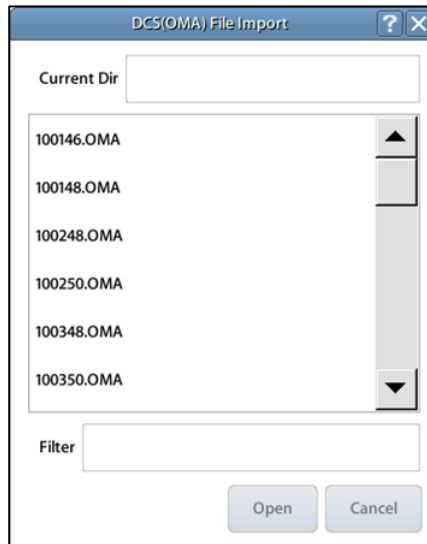

- How to import OMA file
  - ① Create an 'oma' folder in the SD card on your PC.
  - ② Copy OMA files to the 'oma' folder in the SD card.
  - ③ The OMA file extension should be '.oma'.
  - ④ Insert the SD card to the machine.
  - ⑤ Press the L side button for a while.
  - ⑥ Select the file and press the open button.
- Some OMA files may not be fully compatible with our system. So please note that it may not be able to process normally.
- OMA import function is available after being activated in the menu (Ask to the service technician of HUVITZ or the technician authorized by HUVITZ).

## 5.1.2. Job Management

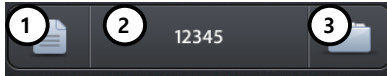

- ① New : Clear current job screen and wait new job.
- ② Job Number : Display current job number (Press the button to change job number).
- ③ Job Manager : Press the job manager button to load a new job or move on to another job.

### NOTE

- Virtual Keyboard  
Use the virtual keyboard to change Job number.



- ① Job number
- ② Backspace
- ③ Apply
- ④ Reset (Cancel) – If you press the button, the value goes back to original value. If you press the reset button one more time, the virtual keyboard disappears.
- ⑤ Move cursor
- ⑥ Cancel (Close) – Close the virtual keyboard without saving changes.

**NOTE**

- Following additional functions are available by pressing 'Job Management' button for a while.
- You can save current job on SD card by pressing 'Job Number' for a while.




- If you append file extension such as ".oma" or ".dxf" at the end of "Job Number", you can export a job to OMA file or CAD file in the "oma/export" or "dxf/export" folder of the SD card (eg. 12345.oma).
- You can request job from the remote edging host pc by pressing the 'Job Manager' button for a while.




- The function of job request works only when communication option on configuration menu is set properly.

## 5.1.3. Frame Information

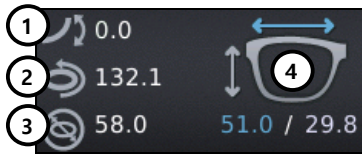

- ① Frame Curve (In case of pattern, input demo lens curve)
- ② Circumference
- ③ Minimum Lens Diameter
- ④ Width/Height


## 5.1.4. Layout Option

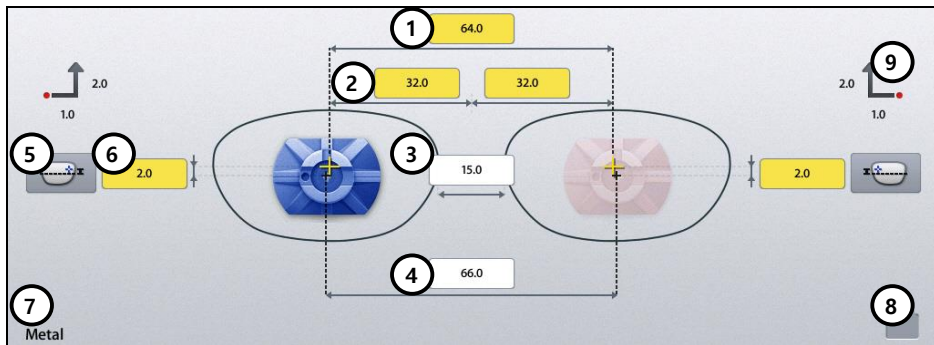


- ① Binocular PD: Input prescribed binocular PD value.
- ② Monocular PD : Input PD values for each side (L/R).
- ③ Bridge Size (DBL): Input value of bridge size.
- ④ FPD: Input value of FPD.
- ⑤ OH Type: Select one of three,  $\Delta Y$ , Box Height, Mix Height.
- ⑥ OH (Optical Height): Input OH value.
- ⑦ Frame Material : Display frame material.
- ⑧ Touch Screen Lock On/Off.
- ⑨ Blocking Offset: Display movement amount of the lens to perform Boxing center blocking.

 **NOTE**

- When you input Binocular PD value, Monocular PD values for each side are automatically calculated and vice versa.
- When you input Bridge Size value, FPD value is automatically calculated and vice versa.
- In case of Two Eye Tracing, the value of FPD will be automatically transmitted and displayed.
- In case of pattern tracing, be sure to change the standard value to the actual Bridge Size.
- In case of One Eye tracing, it will be helpful to use Bridge Size Input function. (It is much easier to measure the Bridge Size.)
- Measure B, not A, for the Bridge Size. Remember that the Bridge Size is not from frame to frame, but from lens to lens.

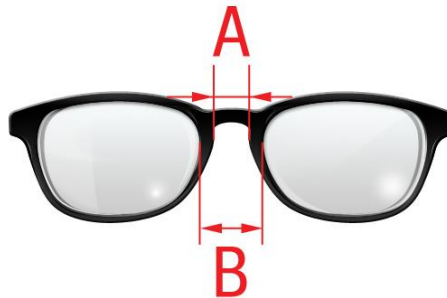


- OH value is recalculated when you change OH type.
- The exact meaning of the mix height and the box height is as following.

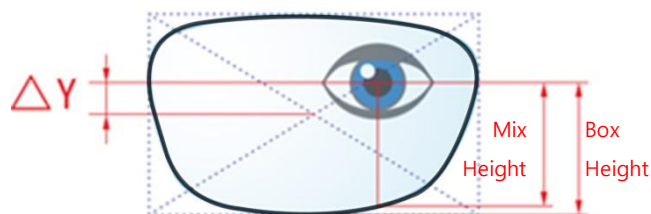


- To enter OH value R/L differently, long press the OH input area.
- It is important to measure the optical height of the patient precisely when edging a progressive lens. Sometimes, too short optical height might cut out the near vision area.

## NOTE


- In layout information, PD, OH, Bridge Size, FPD are mandatory value. The color of input box of these values is changed when you input value.

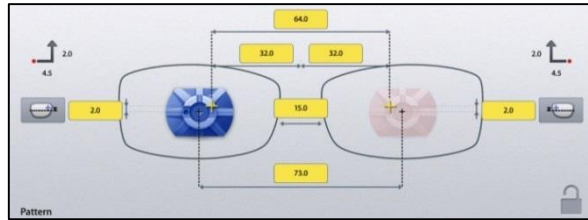


- The box which is not changed from default value is colored yellow. It changes into white when you input value.

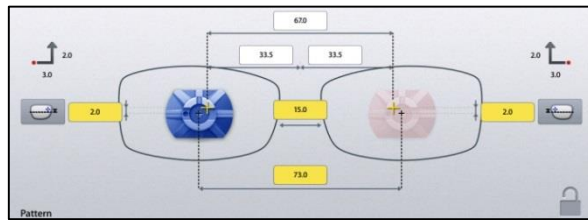

- If frame reader traces shape of frame for both side, FPD value is calculated automatically. Therefore, input boxes of bridge size and FPD are displayed white.

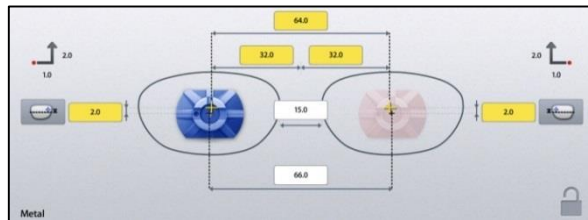

- Warning message pops up if you start blocking with the yellow box remained.

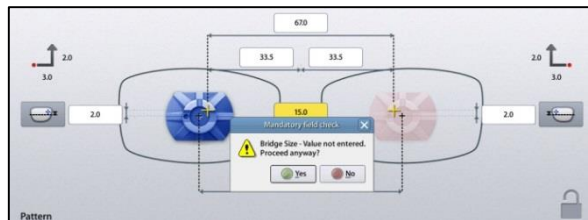

- But, this warning message pops up only when the warning option is turned on in the Preferences menu.


**NOTE**

- Layout information, edging information and lens information can be modified in advance before or while tracing the frame information by frame reader or digital scan. The modified information will be applied to the new job from the frame reader or digital scan.




**NOTE**

- Numeric Keypad  
Click the value you want to change then this numeric keypad will pop up.

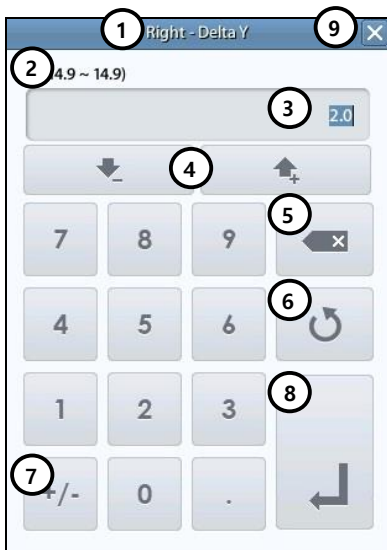

- ① Name of the value
- ② Range of the value
- ③ Value
- ④ Increase/Decrease
- ⑤ Backspace
- ⑥ Reset (Cancel) – If you press it, the value goes back to original value. If you press it one more time, the numeric keypad disappears.
- ⑦ Change Sign
- ⑧ Apply
- ⑨ Cancel – Close numeric keypad without saving the changes. Reset button is helpful when you are in trouble with small size of cancel button.

## 5.1.5. Edging Option

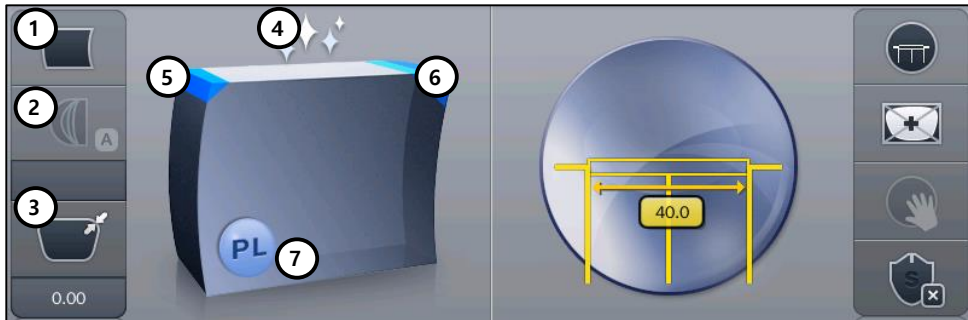

### ① Edging Type



Bevel



Flat



Grooving



Mini Bevel



Asymmetric Bevel



Semi-U Bevel

### ⚠ NOTE

- Available edging types can vary by wheel types of the edger.

#### Normal Type (RPG, RPW)



#### Asymmetric Type (RPA, RPGA)




- Step Bevel edging type is available when the edger type setting is HPE-8000X in the preference menu. (Refer to 'Chapter 6.1 Preference')



 **NOTE**

- You can input additional parameters according to the selected edging type.

- Grooving

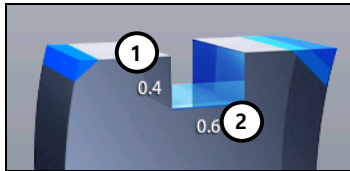


- ① Groove Depth
- ② Groove Width

- Mini Bevel




- ① Bevel Height

- Asymmetric Bevel

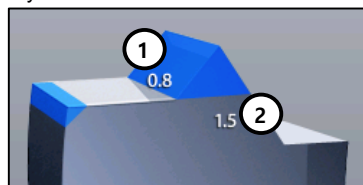

- ① Front Height
- ② Rear Height

- Semi-U Bevel

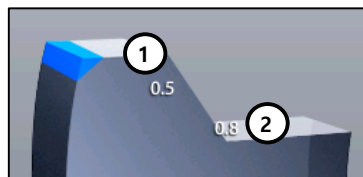

- ① Front Width
- ② Rear Height

- When the edger type setting is HPE-8000X or HPE-8000XN in the preference menu, 'Bevel Height Editor' is provided instead of Asymmetric Bevel / Semi-U Bevel height input.

## NOTE

- After you've selected grooving as the edging type, you can select partial grooving by pressing 'Edging type' button for a while.



- There are four ways of partial grooving.



Partial Grooving  
(Flat+Grooving)



Hybrid Grooving  
(Bevel+Grooving)




Partial Bevel  
(Bevel + Flat)



Dual Grooving

- Relevant icon is displayed at edging type when partial grooving is selected.

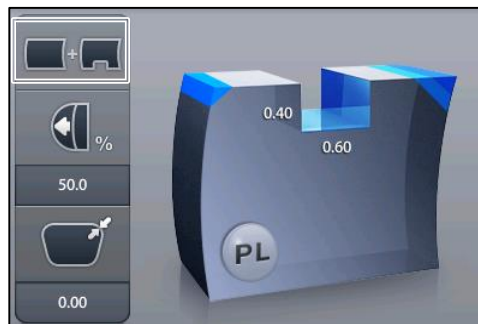

- To cancel the partial grooving mode, press the edging type button and change into another mode.
- Processing with partial grooving needs an editing step of partial grooving for set boundary position after feeling process.

② Bevel/Groove Position




③ Size Adjustment (-2.0 ~ 2.0mm)


④ Polishing



⑤ Front Safety Beveling

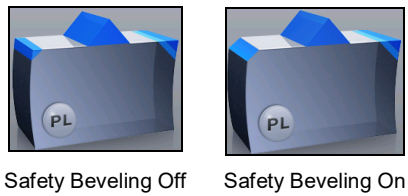


⑥ Rear Safety Beveling

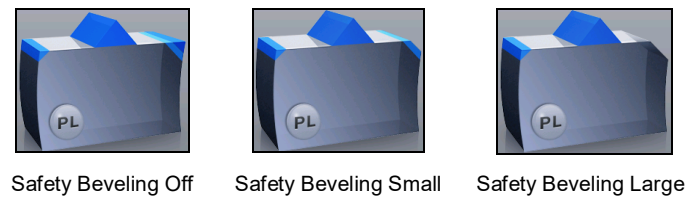

## NOTE

- Polishing / Safety Beveling touch screen area

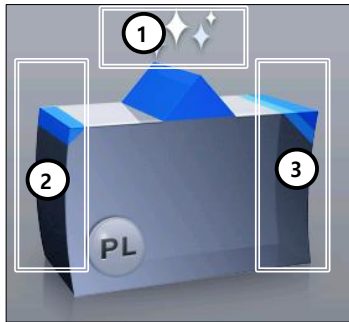

- ① Polishing On/Off
- ② Front Safety Beveling On / Off
- ③ Rear Safety Beveling Off / Small / Large

- When you selected 'Asymmetric Bevel' or 'Semi-U Bevel', rear safety beveling is not available.
- Safety Beveling option is disabled in Hybrid Grooving (Bevel+Grooving) and Partial Bevel (Bevel + Flat).

## ⑦ Lens Material



Plastic



Polycarbonate



Hi-Index



Glass




Trivex®

## NOTE

- If the wheel type of the edger is Asymmetric or Wide, Glass lens cannot be selected.
- Glass lens cannot be processed in the following edging process :  
Polishing, Grooving, Milling, Drilling

**5.1.6. Lens Information**

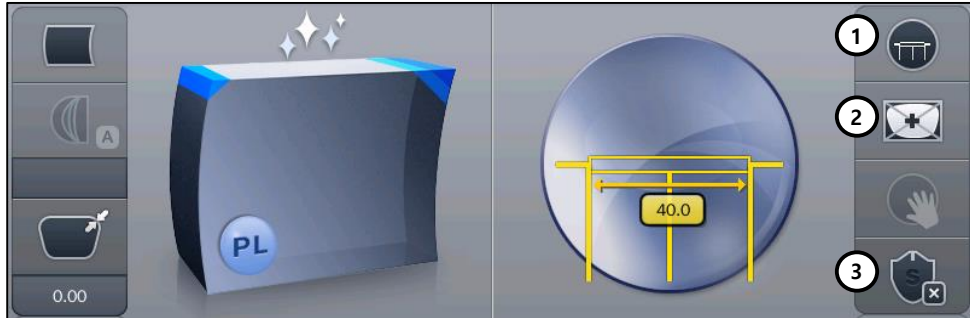

① Lens Type



3-dot marked



Bi-focal  
(Flat)



Bi-focal  
(Round)



Bi-focal  
(Curved)



Progressive

② Blocking Mode



Optical Center

- Set blocking location to the optical center.




Boxing Center

- Set blocking location to the box center. (Frame Center)

③ Safety Mode




Safety Mode Off

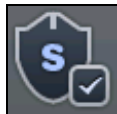

Safety Mode On


## NOTE

- You can change the guideline according to the lens type.
- Press the guideline value box area and enter the value to change.
- Normal lens

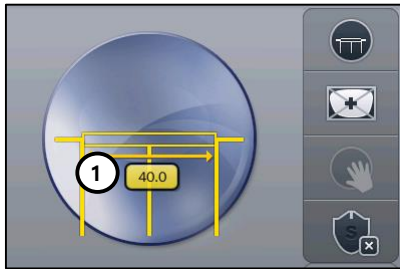


① 3-dot length

- Bi-focal lens

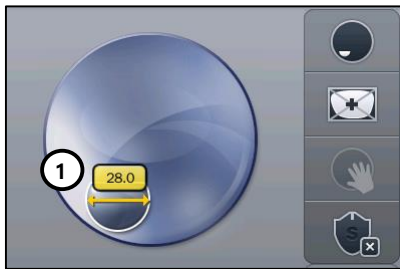

① Near vision area diameter



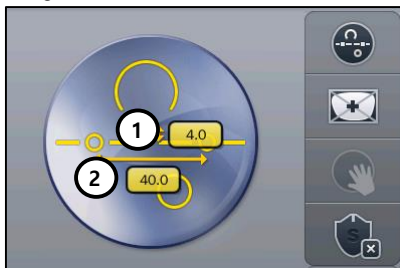



- Progressive lens





① Eye point height

② Axis length


**NOTE**


- Boxing Center is useful for the following cases.
  - When PD is in eccentric position
  - When frame is horizontally wide
  - Super Hydrophobic Lens or Ultra Hi-Index Lens
- When you choose the options for Bi-Focal, Near Vision Shape will be displayed on the screen. If you input the diameter, you can see the frame shape similar to the actual lens shape and check the interference.
- According to the Edging Options, you can control the Beveling and Grooving position.
  -  Ratio(%) from the front side of the lens
  -  Distance(mm) from the front side of the lens
  -  Distance(mm) from the rear side of the lens
  -  Adjust the position according to the lens base curve
  -  Auto mode
  -  Adjust the position manually
- Final outputs according to the edging options are different just like the following pictures.

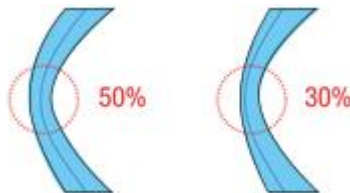


 [Ratio(%) mode from the front side]

- Please be cautious that the thinnest part of a lens may be broken when you use ratio (%) mode from the front side.

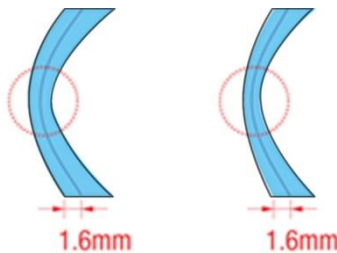

 [Distance(mm) mode from the front side]

- Using the distance (mm) mode from the front side, the operator can designate the specific moving distance. However, if the thickness of the thin part is less than twice the designated value, the machine will process at internal center, which is 50% in ratio.

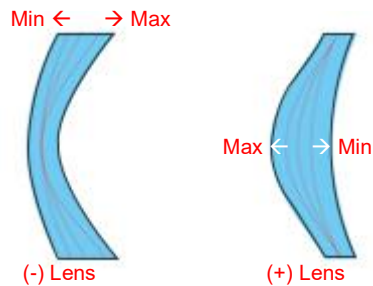


[Base curve mode]

- In the base curve mode, base curves of (+) lenses and (-) lenses exist the opposite side. In this mode, the bevel or grooving line always passes the mid of the thinnest parts, one of which is located at the midst zone of (-) lens and the other is located at the edged zone of (+) lens. And much bigger position value makes the curve closer to the maximum curve.
- The auto mode calculates the most natural fitting position automatically using the lens front/rear curve and the shape of the eyeglasses.
- Frame Material, Lens Material and Lens Thickness may affect the quality of Lens Fitting. If needed, execute the Size Adjustment in advance. Frequently used Adjustment Value can be saved at “Frame Fitting” in “Configuration Menu”.

## CAUTION

- Be sure to carry out the Edging Process in Safety Mode when the lens is fragile or the axis easily moves.
- Grooving and Polishing is not available for the Glass Lens. Be sure the material of lens is matched with the job options selected in the display before starting the edging process.
- Always be sure to use a new Tape or Sticker and remove moisture on the Lens surface.
- The safety beveling for glass lenses should be performed both at front and rear side. Otherwise, it will cause damage to the human skin or eyes.
- To prevent axis shift, you must use stickers provided by lens manufacturers for blocking of hydrophobic lenses. We do not guarantee the quality of the stickers.

### 5.1.7. Function Button

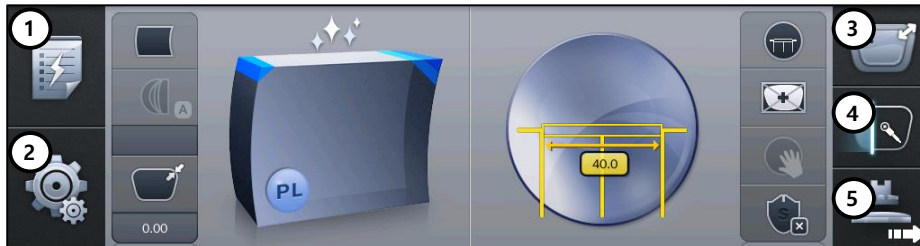


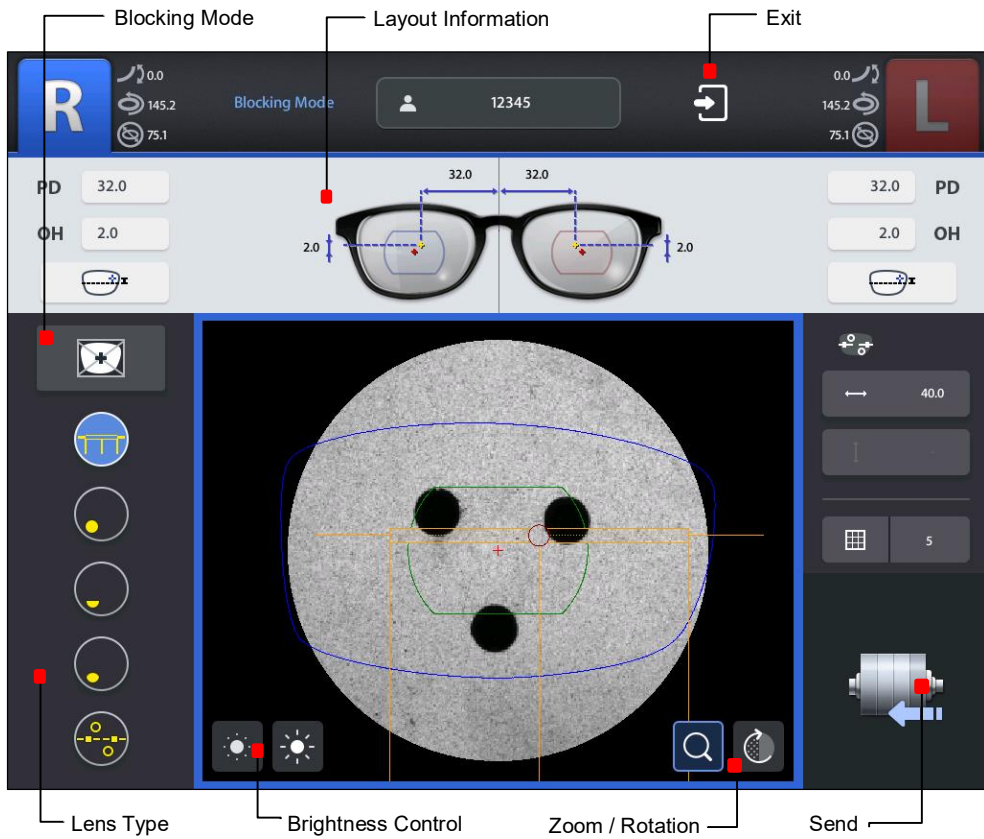
- ① Expert Job Editor
- ② Menu
- ③ Digital Designer (Digital Pattern + Hole Editor)
- ④ Digital Scan
- ⑤ Blocking




The expert user can work more quickly by using 'Expert Job Editor'.

## 5.2. Blocking Screen

Press  button on the main screen to enter blocking screen.




**5.2.1. Layout Information**

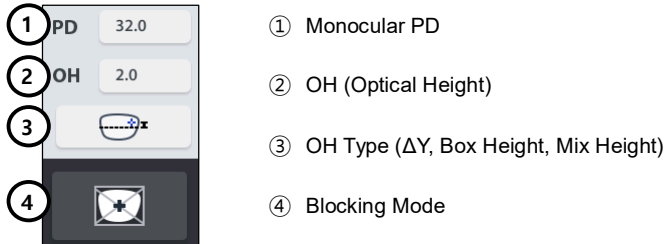


**NOTE**

You can enter PD/OH value directly or fine tune with the +/- buttons.

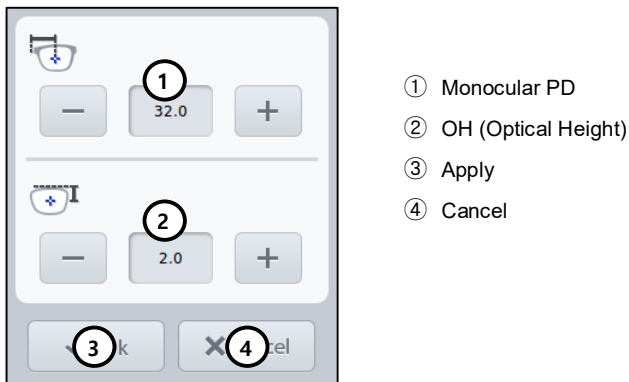


**5.2.2. Lens Type**

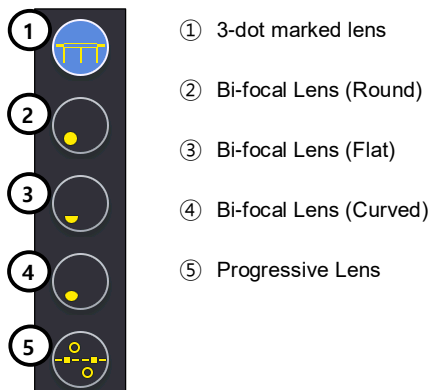

## NOTE

On the right side of the screen you can change the lens guideline and display the grid.

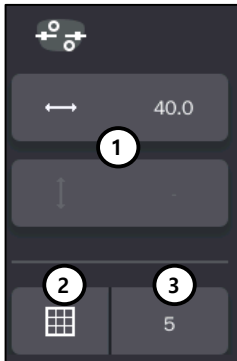

- ① Guideline
- ② Grid On/Off
- ③ Grid space

### 5.2.3. Tool Button



- ① Brightness Control : Low
- ② Brightness Control : High
- ③ Live Image : Normal / Zoom
- ④ Reflector Rotation : On / Off

### 5.2.4. Sending Job




Send : Send the job data to the edger.

### 5.2.5. Exit

Quit the blocking screen and return to the main screen.

**NOTE**

- When the lens type is the progressive lens, you can use laser mark for blocking.
- When the lens type is changed to the progressive lens, the buttons are displayed in the bottom right ( decrease contrast and increase contrast).
- User can recognize the laser mark without any re-marking on the lens. If the laser mark is not clear, adjust the contrast of the screen using and buttons.

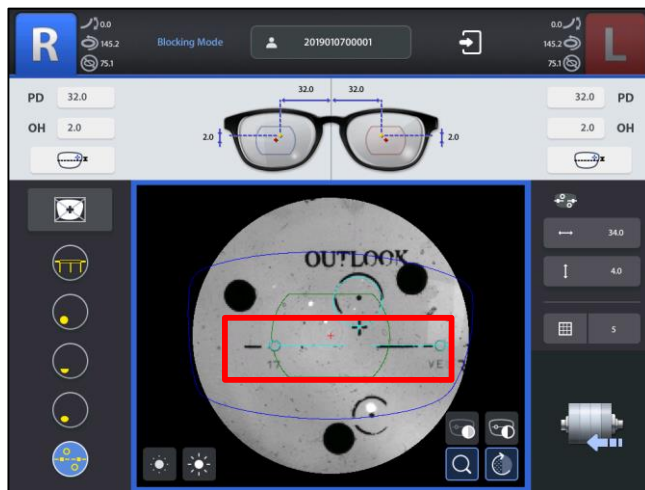


**NOTE**


HPE edge offers different types of lens adaptors. When blocking, use the appropriate lens adaptor for the processing side, lens curve and shape.









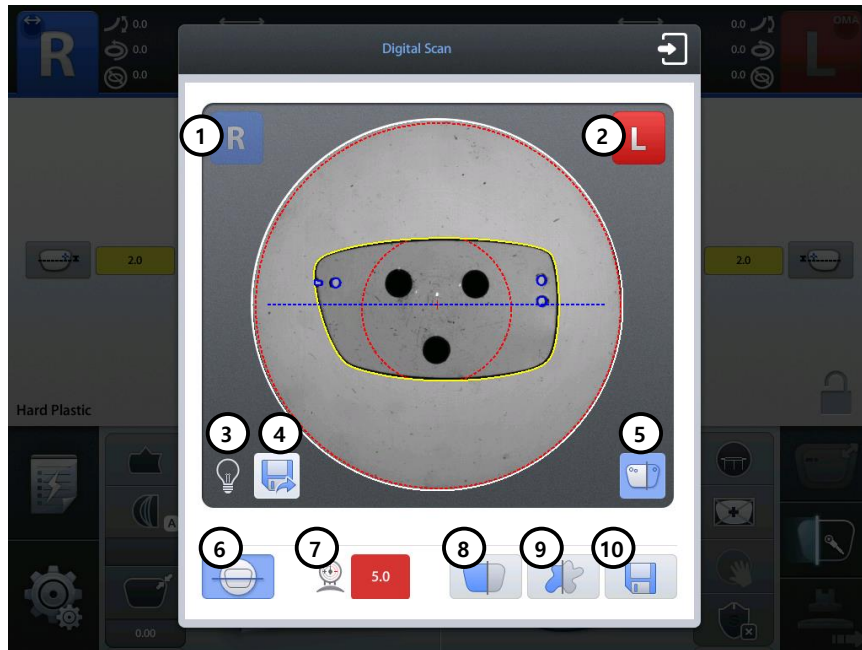



- Color : Blue (Right Side), Red (Left Side)
- Type 1 : High curve small adaptor
- Type 2 : Normal curve small adaptor
- Type 3 : Normal curve large adaptor

## 5.3. Digital Scan

This is for recognizing the frame/holes/slot/notch by scanning demo-lens. To use digital scan, click the button  on the bottom right of the main screen.

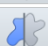


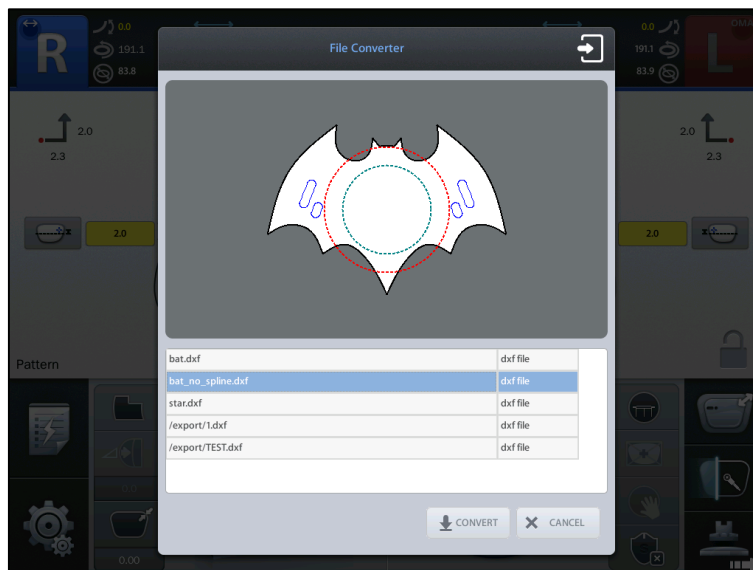
- |   |  |
|---|--|
|  | ① Select right side.   |
|  | ② Select left side.  |
|  | ③ Adjust LED brightness manually.  |
|  | ④ Convert image file(bmp, jpg, png) or CAD File (dxf) to a job file  |
|  | ⑤ Find hole/slot in the digital scanning.  |
|  | ⑥ Set axis of demo-lens manually by dragging the two points. If there is no dragged line, the horizontal line is the axis. |
|  | ⑦ Input demo lens curve for size accuracy  |
|  | ⑧ Start digital scan for normal shape.   |
|  | ⑨ Start scan & cut digital scan for complex shape .  |
|  | ⑩ Save the Digital Scan result to Job Manager.   |

## How to use Digital Scan

- ① Change to Digital Scan lens plate.
- ② Mark 3-dots or draw straight line for axis on the demo-lens. Then, place the lens on the plate.
- ③ Select side.
- ④ Press the start button for Digital Scan.
- ⑤ After Digital Scan is done successfully, frame shape is displayed in yellow color. Otherwise, the frame shape is displayed in red color.
- ⑥ Press the save button to save the scan result to Job Manager.

### NOTE

- Copy image files to SD card in advance to use image file converter.
  - Image file (bmp, jpg, png) : "images" folder
  - CAD File (dxf) : "dxf" folder
- When there is at least one more files in "images" folder or "dxf" folder,  button will appear.



## CAUTION

- The Digital Scan grab the frame shape from the demo lens with high resolution and accuracy. So if the demo lens is different in size or shape from the frame, it'll be reflected in the grabbed frame shape either and the result lens will not make a good fitting in the frame. It's recommended to check and follow the instruction in the below for better fitting.
  - ① When the demo lens is smaller than the frame  
Put size offset value for edging as much as the size difference and cut a lens. If the lens fits bigger than the frame, do retouch in the edger to make it smaller.
  - ② When the edge of the demo lens is bumpy and not smooth  
Make the edge smooth by sandpapering or filing before scanning it. Then you can get more even and smoother frame shape by the Digital Scan
  - ③ When the shape of demo lens is not exactly matching that of the frame  
Use the Digital Pattern function to modify and correct the shape before you make a cut for the scanned shape.

### 5.4. Job Manager

The screenshot shows the Job Manager application window. At the top, the title bar reads "Job Manager". Below the title bar is a toolbar with several icons: a database icon, a refresh icon, a circular arrow icon, a download icon, and a magnifying glass icon. A red square highlights the "Job selection tab" icon in the toolbar. In the top right corner of the window, there is an "Exit" button with a red square next to it. The main area of the window contains a table with the following data:

	Job Number	Type	Reading	Frame Type	FPD	DBL	Time
<input type="radio"/>	2016102700011		Both	Pattern	75.0	17.0	2016-10-27 14:57:13
<input type="radio"/>	2016102700010		Both	Metal	68.0	17.0	2016-10-27 14:49:51
<input type="radio"/>	2016102700009		Both	Metal	68.0	17.0	2016-10-27 14:49:48
<input type="radio"/>	2016102700008		Both	Metal	68.0	17.0	2016-10-27 14:49:45
<input type="radio"/>	2016102700007		Both	Metal	68.0	17.0	2016-10-27 14:49:41
<input checked="" type="radio"/>	2016102700006		Both	Metal	68.0	17.0	2016-10-27 14:49:38


At the bottom of the window, there is a panel with four function buttons: a folder icon, a document icon, a magnifying glass icon, and a trash can icon. A red square highlights the magnifying glass icon. Annotations with lines point to the "Job selection tab", "Exit", "Job list", and "Function button".

## 5.4.1. Job selection tab



- ① Waiting Job : Newly received Job data from the frame reader or Digital Scan.
- ② Working Job : Blocking job or Editing job
- ③ Completed Job : Blocked Job
- ④ Saved Job List : Job data saved in SD card.
- ⑤ Internal Data : Internal job data for testing

## 5.4.2. Function Button

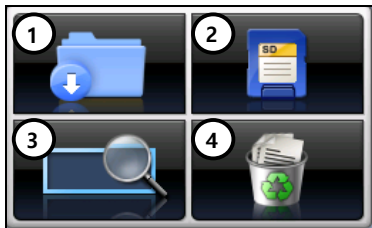

- ① Open : Load the selected job
- ② Save : Save the selected job to SD card
- ③ Search : Search Job number
- ④ Delete : Delete selected job

### NOTE

- Save function is only available on Waiting/Working/Completed list.
- If the power of the system is turned off, all job data on Waiting/Working/Completed list are deleted. So if needed, be sure to save the job data to SD card before turning off the system.
- Internal data cannot be deleted.
- If you delete a job in the saved job list, it will be permanently deleted from the SD card, so be careful.

### 5.4.3. Waiting Job

	① Job Number	② Type	③ Reading	④ Frame Type	⑤ FPD	⑥ DBL	⑦ Time
<input type="radio"/>	2016102700011		Both	Pattern	75.0	17.0	2016-10-27 14:57:13
<input type="radio"/>	2016102700010		Both	Metal	68.0	17.0	2016-10-27 14:49:51
<input type="radio"/>	2016102700009		Both	Metal	68.0	17.0	2016-10-27 14:49:48
<input type="radio"/>	2016102700008		Both	Metal	68.0	17.0	2016-10-27 14:49:45


- ① Job Number: Displays the Job Number.
- ② Type: Displays the Edging Type with icon.
- ③ Reading: Displays the Tracing State with icon.
- ④ Frame Type: Displays the Frame Type.
- ⑤ FPD: Displays the FPD value.
- ⑥ DBL: Displays the DBL value.
- ⑦ Time: Displays the job received time.

### 5.4.4. Working Job

	Job Number	Type	Reading	① Blocking	② Time
<input type="radio"/>	2013062600001		Both	R L	2013-06-26 15:36:22

- ① Blocking : Displays the progress of blocking.
- ② Time : Displays the job start time.

## 5.4.5. Completed Job

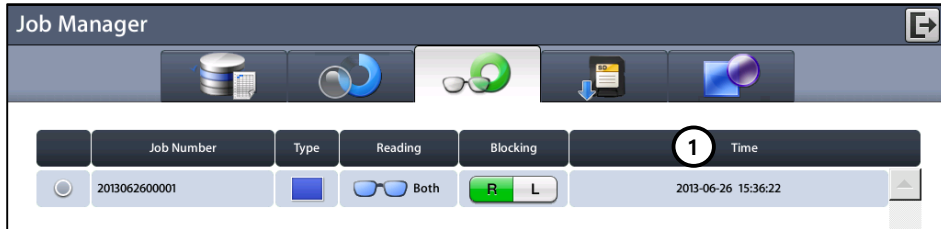

- ① Time: Displays the job finish time.

### NOTE

- You can check current processing state with the following icons.



### 5.4.6. Job Search

You can search for a job on the job list.

- ① Start 'Job Search Mode' by pressing the search button.
- ② Enter the job number you want with the virtual keyboard.
- ③ Check the list.



Searched item display

	Job Number	Type	Reading	Frame Type	FPD	DBL	Time
<input type="radio"/>	2013061700004		Both	Pattern	73.0	15.0	2013-06-17 18:52:49
<input type="radio"/>	2013061700003		Both	Metal	66.0	15.0	2013-06-17 18:52:46
<input checked="" type="radio"/>	HUVITZ_123		Both	Hard Plastic	71.5	21.1	2013-06-17 18:52:29
<input type="radio"/>	2013061700002		Both	Pattern	55.0	15.0	2013-06-17 18:52:16
<input type="radio"/>	2013061700001		Both	Metal	64.1	19.0	2013-06-17 18:01:12

Keyword:HUVITZ  
 Find Prev Find Next  
 FPD : 71.51  
 DBL : 21.10  
 Hard Plastic  
 Single Vision


PL


Prev/Next Search button

**NOTE**

- You can search with only few letters, not the whole words. For example, you can search the Job Number 'HUVITZ\_123' only with 'HUVITZ' or '123'.
- You can look at the other research results with 'Next' and 'Prev' button.
- You can finish the Job Search mode by pressing the search button one more time. The search button shows the state change of this mode.

## 5.5. Digital Pattern

The “Digital Pattern” modifies the lens shape in left/right, top/bottom as well as rotation and optimizes the fitting for Rimless and Semi-Rimless. To use Digital Pattern, press  button on the bottom right of the main screen and then select the Digital Pattern tab at top left of the screen.

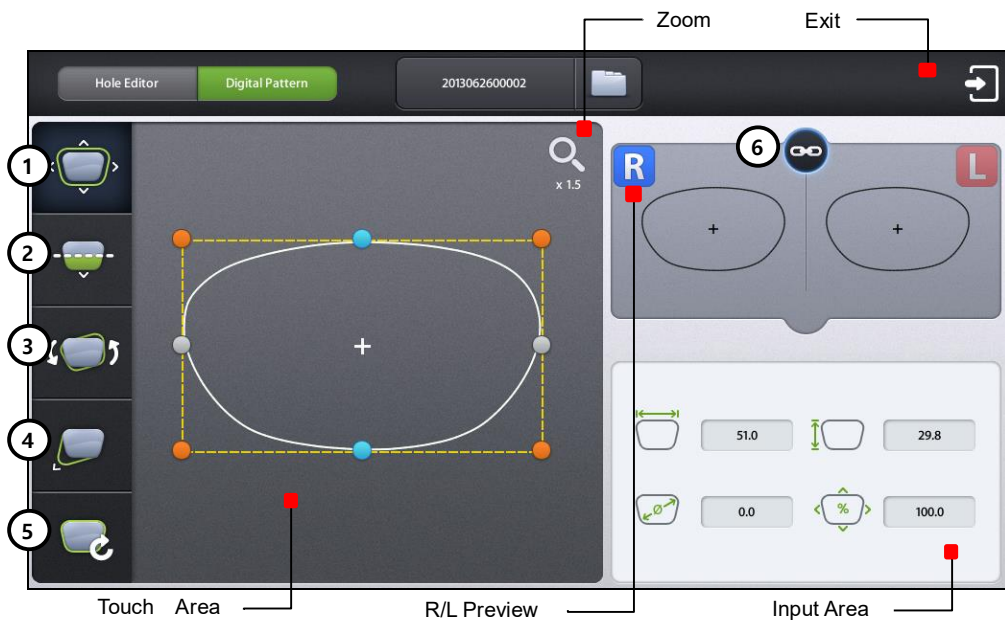


- ① Enlargement/Reduction
- ② Near Vision Area Enlargement/Reduction
- ③ Rotation
- ④ Partial Modification
- ⑤ Reset
- ⑥ R/L Sync

## 5.5.1. Enlargement/Reduction


Lens shape can be modified both horizontally and vertically.

### ■ Touch Interface

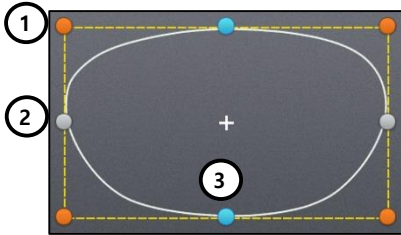


To modify lens shape, make use of the 8 touch points around the lens shape.


- ① ● Horizontal + Vertical
- ② ● Horizontal
- ③ ● Vertical

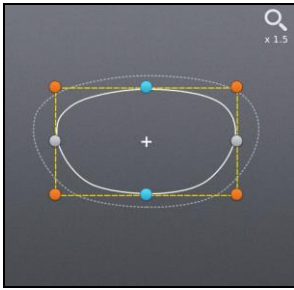

[Horizontal + Vertical]

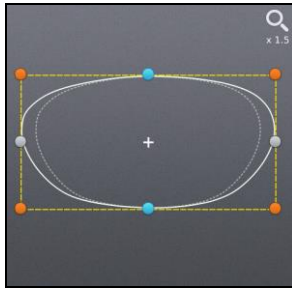


[Horizontal]

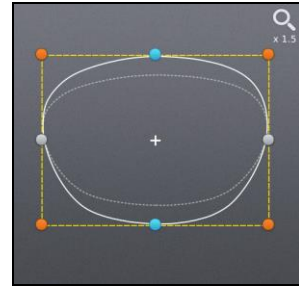


[Vertical]

### ■ Value Input

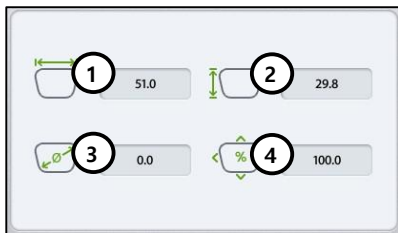

You can modify the lens shape by entering the values directly in the Input Area.

- ① Width
- ② Height
- ③ Size (mm)
- ④ Percent (%)

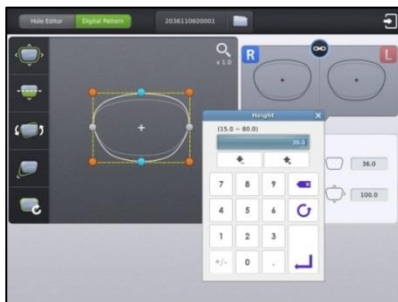


After selecting the item to change, enter the value using the numeric keypad.

### 5.5.2. Near Vision Area Enlargement/Reduction


Lens shape can be changed in four directions (Bottom - Near Vision Area, Top, Left, Right).

#### ■ Touch Interface

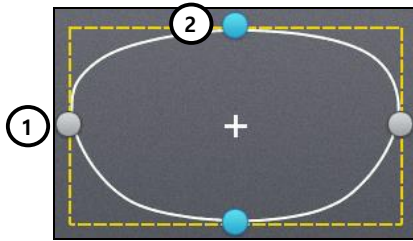


To modify lens shape, make use of 4 touch points around the lens shape.


- ① ● Left or Right
- ② ● Top or Bottom

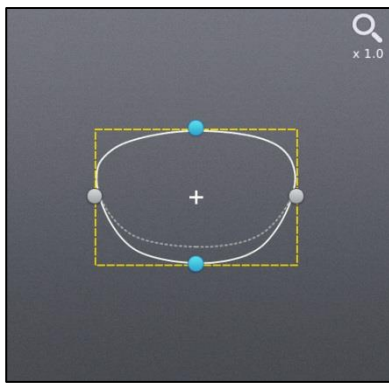


[Near Vision Area (Bottom)]

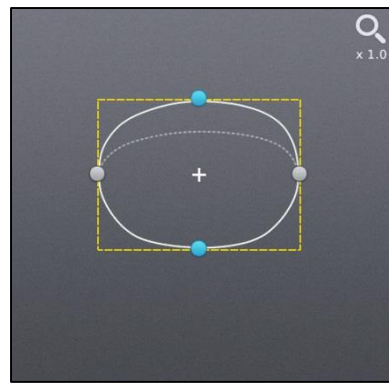

[Top]

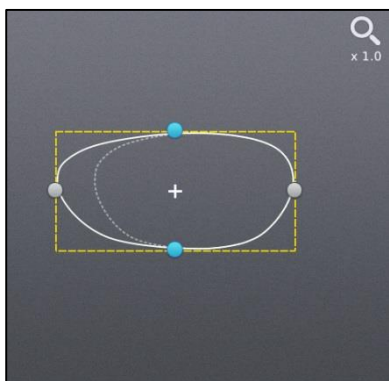


[Left]

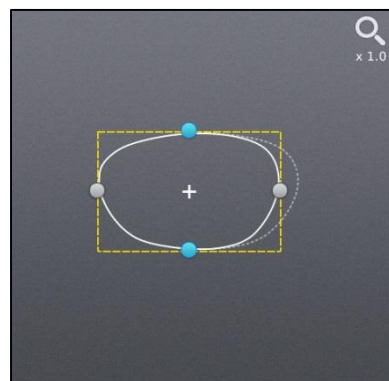

[Right]

## Value Input

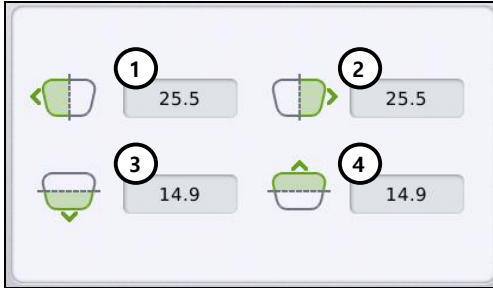

You can modify the lens shape by entering the values directly in the Input Area.

- ① Left
- ② Right
- ③ Near Vision Area (Bottom)
- ④ Top




After selecting the item to change, enter the value using the numeric keypad.

## 5.5.3. Rotation


Lens shape can be rotated.

## Touch Interface

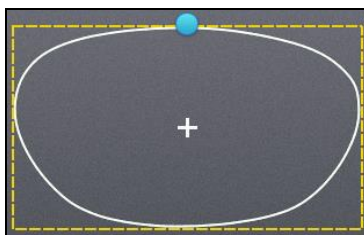


To rotate lens shape, touch and drag the touch point on top of the lens shape.

- Touch Point

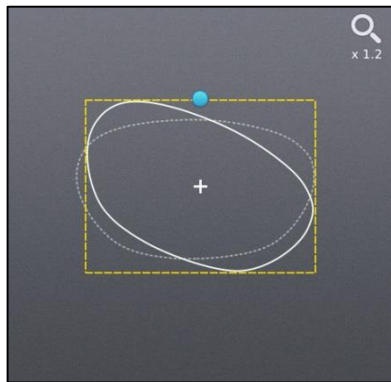


[Clockwise]

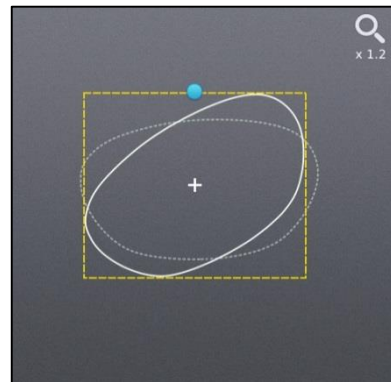

[Counterclockwise]

## Value Input

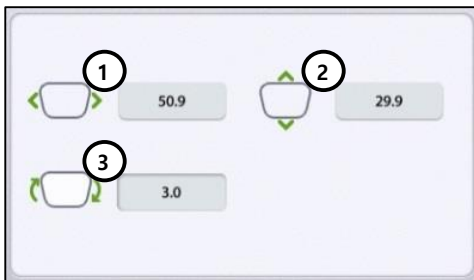

You can modify the lens shape by entering value directly in the Input Area.

- ① Width
- ② Height
- ③ Angle (Clockwise, 0 ~ 360)

➤ Function of ① and ② are same with that of Enlargement/ Reduction mode.




After selecting the item to change, enter the value using the numeric keypad.

## 5.5.4. Partial Modification

Lens shape can be modified partially.


### ■ Touch Interface

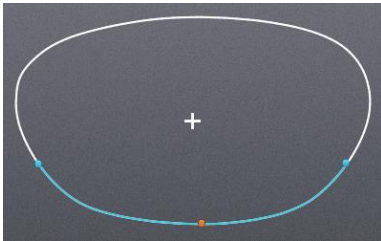


Change modification area by touching any point on lens shape. The point becomes the center of the modification area.

To adjust modification range, touch and drag one of 2 touch points around the selected lens shape (blue).


- Range adjustment point
- Edit Position (Most changing position)
- User touch location

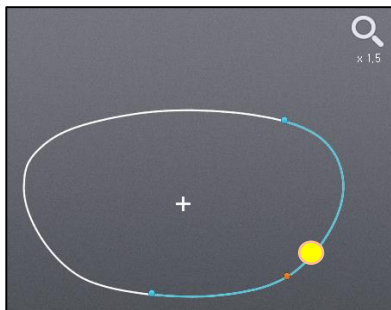


[Bottom Right]

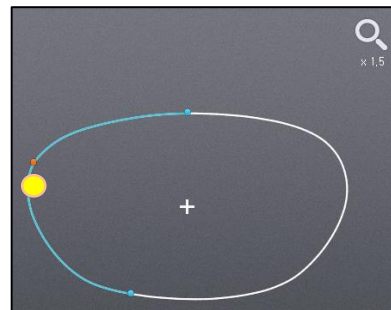

[Left]

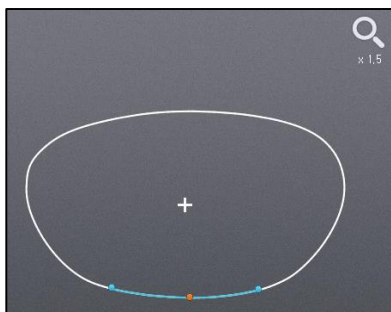


[Modification area reduction ]

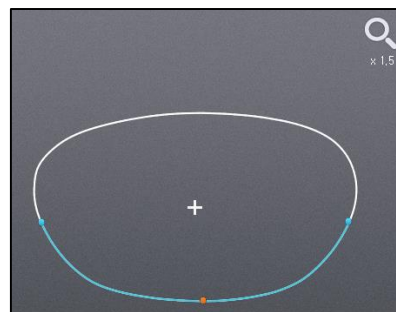


[Modification area enlargement]

**■ Lens Shape Modification**

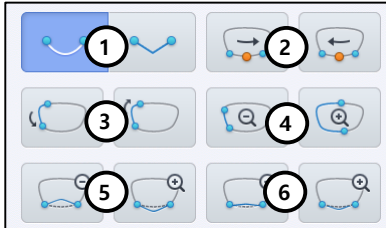


You can modify the lens shape with Input button.

- ① Edit mode (Curved or Linear)
- ② Edit Position Change
- ③ Position change (counter clockwise, clockwise)
- ④ Range change (enlargement, reduction)
- ⑤ Wide Partial Modification (concave, convex)
- ⑥ Partial Modification (concave, convex)

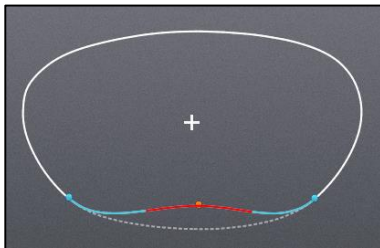


During partial modification, the modified part where is not able to process physically is colored red. And when closing the digital pattern, the shape can be modified to edging enabled shape intelligently.

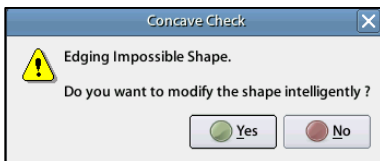


**⚠ NOTE**


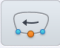
- The icon of end point changes to be enlarged when end point is positioned on the corner of the lens shape and as the end point close to the corner, the point snapped to the corner automatically.
- In the linear edit mode, the follow editing is possible by moving the edit position to one of ends point and moving the other end point to the corner of the lens shape. The enlarged part follows the out part of the edit position(red box).

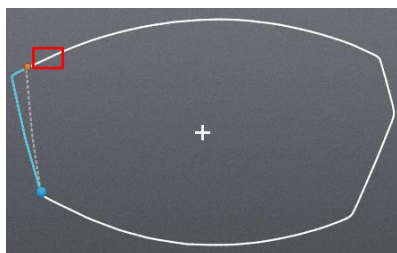

[Convex Edit]

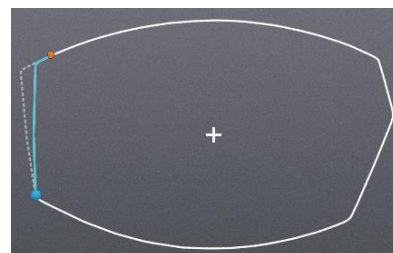


[Concave Edit]


- 
- The Edit position can be switched to the center or the other end point quickly by holding edit position change button (   ).

## 5.5.5. Toggle R/L Sync

Function for R/L Sync.

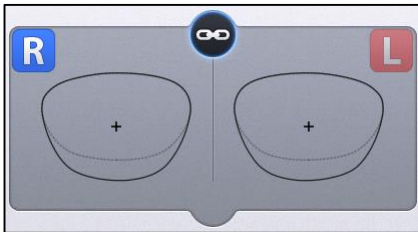

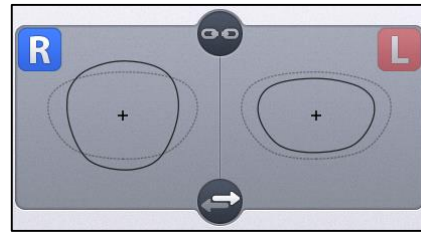


[R/L Sync On]



[R/L Sync Off]

### NOTE


- “Copy” button (  ) is available when R/L sync option is turned off. Shape of the current side is copied to the other side by pressing this button.
- Press  or  button to change the side.

### 5.5.6. Reset

Recover original lens shape.


### 5.5.7. Exit

Quit digital pattern and return to the main screen. To apply modified lens shape, Press the “Apply” button on the confirm dialog box.

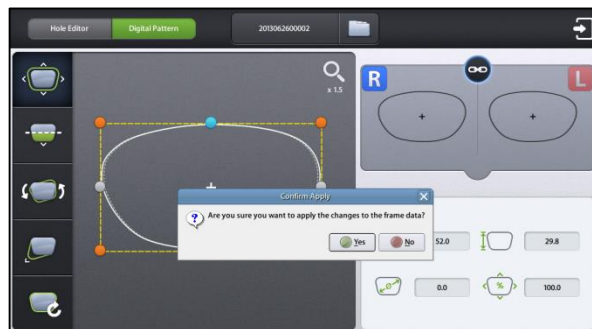


### NOTE

- You can modify half-framed lens or rimless lens conveniently by using the digital pattern.
- The digital pattern is useful for the bi-focal lens or the progressive. Sometimes too small frame shape might cut out the near vision area. In this case, the digital pattern provides the enlargement of the frame shape.
- When digital pattern is applied, FPD or bridge size is recalculated according to the digital pattern applying option on Configuration menu.
  - Fix FPD – Fixing FPD, re-calculate bridge size depending on modified lens shape
  - Fix Bridge Size – Fixing bridge size, re-calculate FPD depending on modified lens shape
- Be sure to check optical center position and PD/OH value after digital pattern is applied.
- Press the Zoom icon to change zoom ratio of the lens shape.

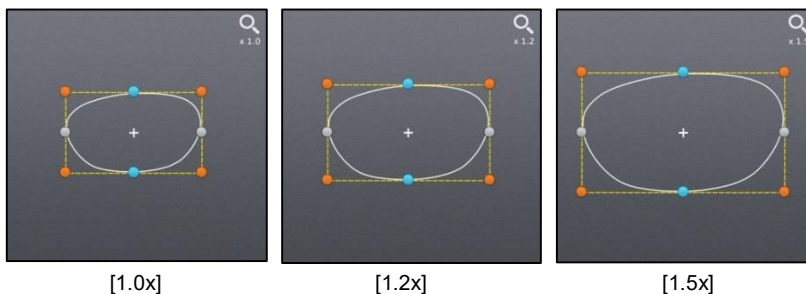

## 5.6. Hole Editor


This screen is for the edition of hole position for drilling. To use the hole editor, click the button(  ) on the main screen right below, and then select the hole editor button in the upper left.



### NOTE


- To perform a drilling, the blocking mode must be set to the center of the frame (box center).
- If you add a slot or hole on the hole editor, the blocking mode is changed to the center of the frame (box center) automatically. In this case, the blocking mode is fixed to the center of the frame (box center) and cannot be changed by the user.
- If all of the holes / slots are removed on the hole editor, the user can change the blocking mode.



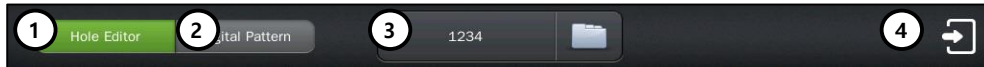

### 5.6.1. Upper Side of Hole Editor

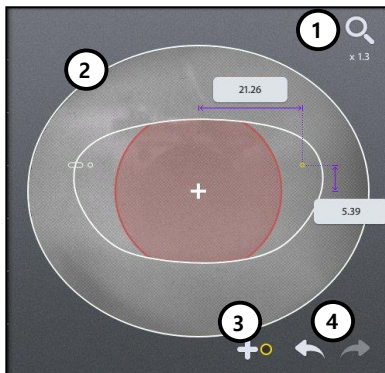

- ① Hole Editor :  
Starts the hole editor. If the button is green, hole editor is now available.
- ② Digital Pattern :  
Starts the digital pattern.
- ③ Job Number :  
Displays the job number of currently working job. You can modify the number by touching here.
- ④ Exit : Terminates the hole editor.


### 5.6.2. Working Area




- ① Magnification: You can select the magnify power either 1.3 times (default) or 5 times (magnified). Partial image of the frame shape is displayed on working area as you magnify the screen. If you want to see other parts of the image, please touch the red circle on preview.
- ② Working Area: Working area displays the shape of frame and holes/slots. The hole/slot in yellow is the selected one. The orange holes/slots mean the same group with selected one. The selected hole/slot is displayed with their coordinates. The user can move the selected hole/slot by touching the LCD or inputting their coordinates directly.
- ③ Add hole/slot: The user can add hole/slot by pressing  button or pre-defined hole/slot set by pressing preset buttons (  ).
- ④ Undo() / Redo(): Adding, deleting, moving and changing of hole/slot

**NOTE**

Elements required for drilling are displayed on working area. The figure below describes each element.

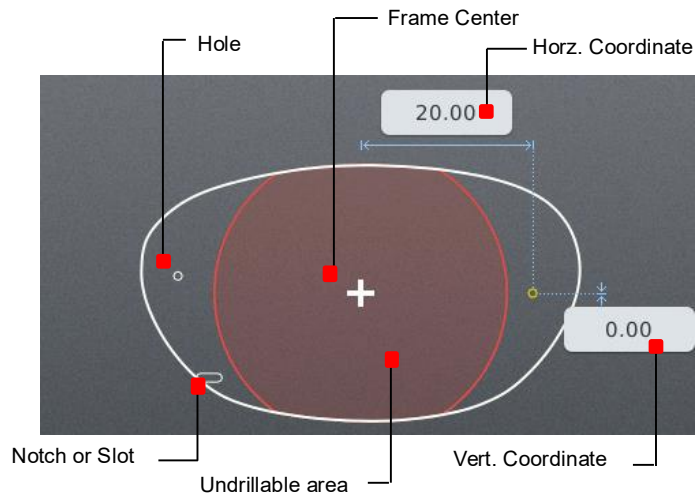

### 5.6.3. Preview

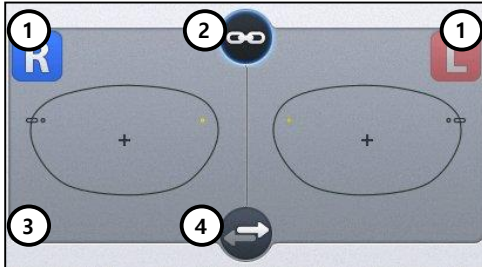

① Select Right/Left:

You can change the selection of the left/right. When the camera mode is set to 'still image', you cannot change the side (L/R) because the frame is not identical with the image. You must change the camera mode from still image to live image first to change the side selection.

② Right/Left synchronization:

The synchronization of the right and left side can be turned on or off. Properties or changes of the selected side are identically applied to the other side, when the synchronization is on.

③ Preview:

You can see the shape of holes / slots on both sides at once.

④ Copy to the other side:

You can copy the holes / slots of the selected side to the other side. The modifications which were performed without the synchronization can be applied to the other side by this function. When you execute this function, all of the holes /slots on the other side will be deleted first.

## 5.6.4. Function Buttons



### ① Hole / Slot Detection:

You can use this button when you want to add holes/slots after the frame reader has read a demo lens. If the camera is on live image mode, you can start detection by pressing this button. If the camera is on still image mode, you can change the mode to live image and start detection by pressing this button.

### ② Switch Camera Mode:

You can switch the camera mode. When you switch the mode from live to still, shape of the demo lens is read and shapes on LCD and camera are matched to each other. When the camera is the state of the live image, it works not only to switch the camera state the still image, but also to make the frame shape on the LCD overlap to the frame shape on the camera image.

### ③ Hole / Slot Easy Moving :

You can move the selected holes/slots. This function provides you precise movement.

### ④ Grouping :

You can select more than a hole/slot and bind them as a group. Select holes/slots which have no group or belong to different groups, and press this button. If you want to cancel the group formation, select one or more holes/slots within the group and press this button.

### ⑤ Preset :

This provides the function to manage preset configuration.

### ⑥ Mirroring :


Selected hole / slot is copied to the symmetric position based on the center of frame. If the selected hole / slot belongs to a group, all holes / slots in that group are copied, and they will form a new group.

### ⑦ Delete Hole / Slot :


This function deletes the selected hole / slot. If it belongs to a group, all of the holes / slots in that group are also deleted.

**NOTE**


- Demo lens for switching camera mode and detection of hole/slot must have an axis indicator. There are 2 ways to indicate the axis. ① To draw a straight line along the axis. ② To mark the 3 dots along the axis. If there is no axis indicator, the horizontal line on the camera image is regarded as the axis of demo lens. Red lines on the below figures respectively represent the axis recognized.

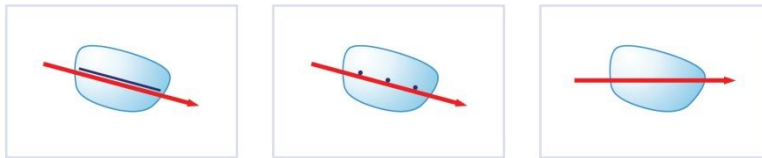

- When you press the Hole / Slot Easy Moving button, following screen will pop up. You can use this to move the hole / slot.

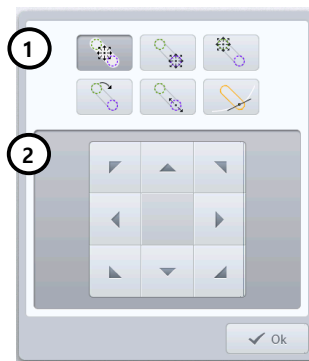


- ① Select moving type: You can select moving type. When the selected one is a hole, it supports just one moving type. When the selected one is a slot, it supports 6 types of moving.
  - ② Move hole / slot: It provides moving buttons according to the selected moving type. How to move the selected hole / slot according to the moving type is explained below.
- The slot icon on the buttons of moving type represents the start(○) and end(○) position like below figure.

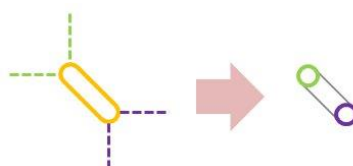


- The details of how to move the hole / slot by moving type are like below.




: This provides the function of parallel movement. The movement of hole / slot is like below.

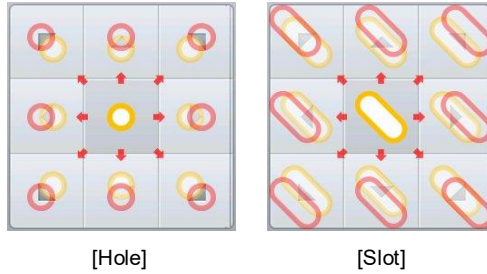


: (Slot only) This provides the function to move the start position of a slot.

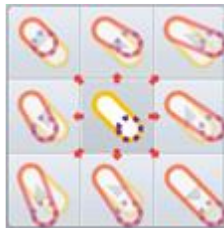


: (Slot only) This provides the function to move the end position of a slot.

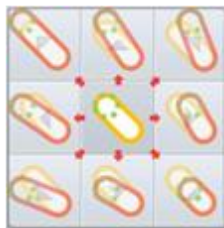


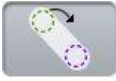


: (Slot only) This provides the function to rotate a slot.

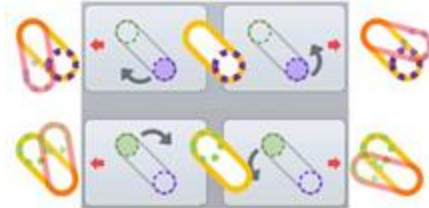

: (Slot only) This provides the function to change the length of a slot.

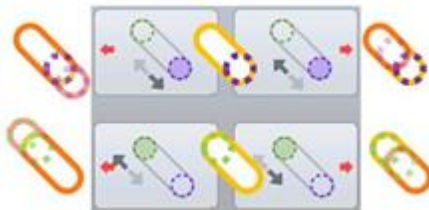


: (Slot only) This provides the function to change a slot to a notch and to move the notch. The slot will be moved to the edge of a frame close to current position, and become a notch. The slot can be moved with the notch remained by using the moving buttons. The up and down buttons are for parallel movement of the slot along with the frame. The left and right buttons are for length modification of the slot lying across the frame. The below figure is the example of the notch moving. (The thick solid black line is the frame edge.)

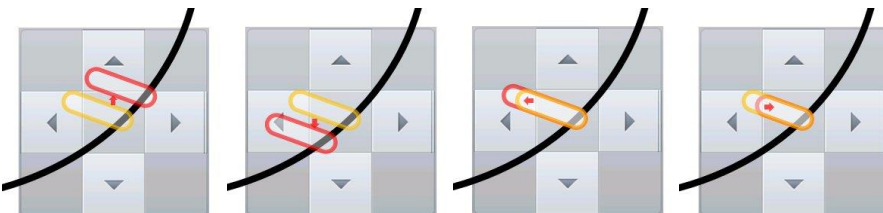

## NOTE


- On the preset, there are three functions.

- Save the hole / slot data: You can save the hole / slot data of the current job to the SD card.

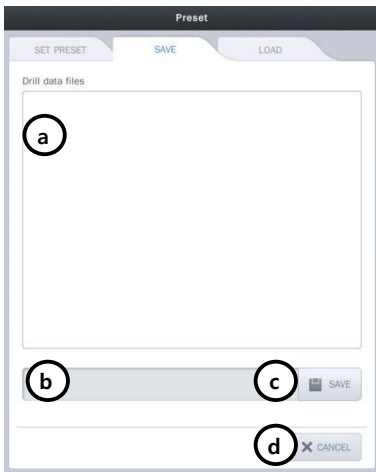


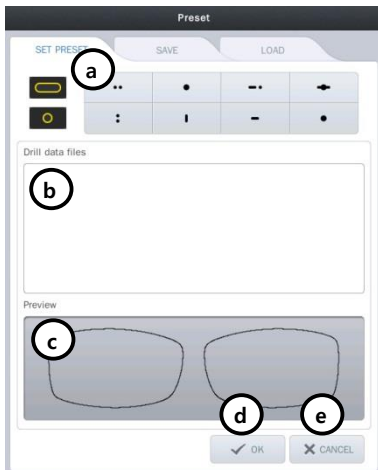
- List of hole / slot data :  
This displays a list of hole / slot data in the SD card.
- File name :  
You can set the file name of the hole / slot data to be saved.
- Save :  
You can save the hole / slot data by pressing this button.
- Cancel :  
Exit the preset configuration.

- Load the hole / slot data: You can load the hole / slot data from the SD card and include them to the current job.

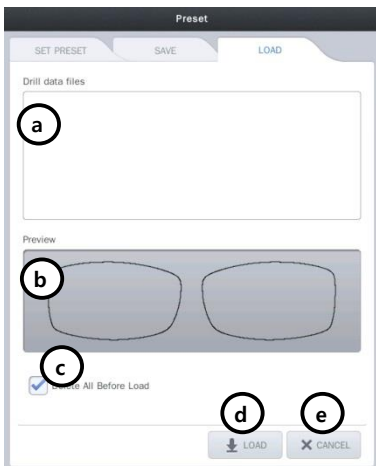

- List of hole / slot data :  
This displays the list of hole / slot data in SD card.
- Preview :  
This displays the shape of the selected hole / slot data.
- Load option :  
You can choose whether to keep or to delete the existing hole / slot data before adding the loaded data. The default value for this option can be specified on the preference.
- Load :  
You can load the hole / slot data by pressing this button.
- Cancel :  
Exit the preset configuration.


- ③ Set preset: You can set the 8 preset buttons. (  ). You can put hole/slot saved in SD card to each button. These 8 presets can be used to add the favorite set of the hole / slot data to the current job, easily.



- a. Select the preset button :  
Select the preset button to apply the hole / slot data. And then choose hole / slot data you want.
- b. List of hole / slot data :  
This displays the list of hole / slot data in SD card.
- c. Preview :  
This displays the shape of the saved hole / slot data.
- d. OK :  
Applies all of the changings.
- e. Cancel :  
Exit the preset configuration.

## NOTE


The mirroring is a function that copies the selected hole / slot to the opposite side. The distance between mirrored hole/slot and the frame edge is same with the distance between original one and the frame edge. In case of slot, the distance between the start point of a side (closest point to the frame) and the frame must be identical with the distance between the end point of the other side (closest point to the frame) and the frame (The two points must be symmetric). If the selected hole / slot belong to a group, all holes / slots are also mirrored, and they will form a new group.

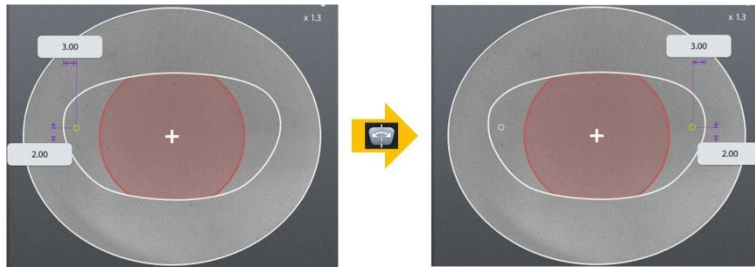

[Mirroring the hole]



[Mirroring the slot]

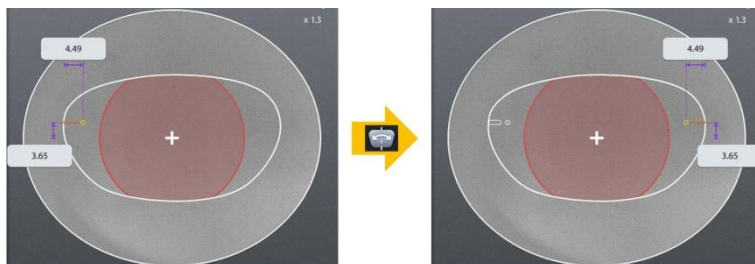

[Mirroring the group of hole / slot]

### 5.6.5. List of Hole/Slot

It displays the hole / slot list of the current job. You can change the position of the hole / slot by pressing the coordinates displayed on the list.

	Gr	ID	X	Y	EX	EY	Ref X	Ref Y	ATTR	Leader	Link
①		1	5,87	17,00			—	—	●	☐	-
②	1	3	4,00	22,00	6,00	22,00	0,00	5,00	—	☐	-
③	1	2	5,87	17,00			—	—	●	☐	-

① Select the hole / slot :

You can select the holes / slots to make a group by pressing here. The green circle is the selected one.

② Group ID :

It displays the group id that the hole / slot belongs to. But if the hole / slot does not belong to a group, nothing will be displayed here.

③ Hole / slot ID :

It displays the hole / slot ID for classifying each other.

④ Change how to display the X coordinate :

You can change the way X coordinate is displayed. Followings are the three options.



Based on the center of frame (Distance from the center of frame)



Based on the box (Distance from the box which is surrounding the frame)



Based on the frame edge (Distance from the edge of frame)

⑤ Change how to display the Y coordinate :

You can change the way Y coordinate is displayed. Followings are the two options.



Based on the center of frame (Distance from the center of frame)



Based on the bottom of box (Distance from the bottom of box)

⑥ Change how to display the X coordinate of a slot

You can change the way X coordinate of a slot is displayed



Based on the option by ④



the length of a slot in X direction



the slope of a slot

⑦ Change how to display the Y coordinate of a slot

You can change the way Y coordinate of a slot is displayed



Based on the option by ⑤



the length of a slot in Y direction



the length of a slot

⑧ Relative distance in X direction

Distance in X direction from the reference hole/slot. Displays – value in the left side or + value in the right side based on the reference position. In case of the reference hole/slot, displays . The reference positions are separated in the left side and right side based on box center and the relative distances are displayed when the number of hole/slot is large than 2 in one side.

⑨ Relative distance in Y direction

Distance in Y direction from the reference hole/slot. Displays – value in the down side or + value in the upper side based on the reference position. In case of the reference hole/slot, displays .

⑩ Hole/Slot



Hole



Slot/Notch

⑪ Display the side of hole/slot and change the reference

Displays whether the hole/slot is position in left or right side base on the box center.

When there are more than 2 Holes/Slots in one side, the closest Hole/Slot to lens shape becomes the reference. When you touch the other hole/slot icon, the hole/slot becomes the reference.

Touch the button , the list of hole/slot will be sorted by the side



Reference hole/slot in the left side based on the box center



Hole/Slot in the left side based on the box center




Reference hole/slot in the right side based on the box center




Hole/Slot in the right side based on the box center

 **NOTE**

- Display options for X coordinate

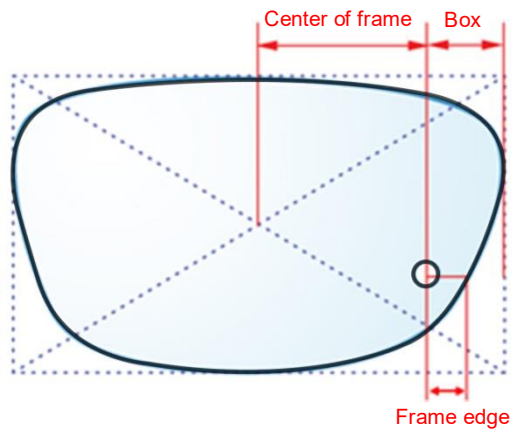


- Display options for Y coordinate

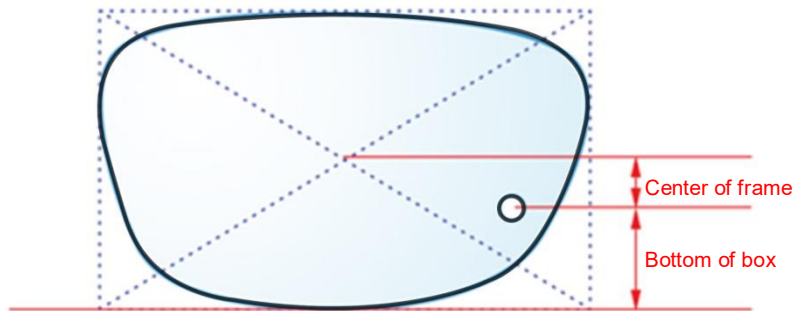

## 5.6.6. Property of Hole/Slot

It displays the properties of the selected hole / slot.

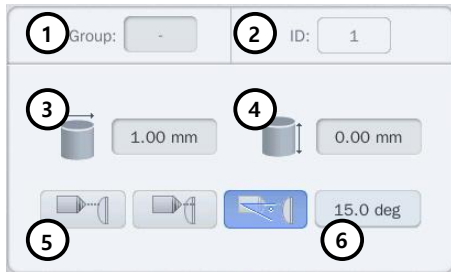

① Group : If selected hole / slot belongs to a group, it displays the group id. If selected hole / slot does not belong to a group, it displays with the '-'.  
② ID : It displays the serial number of the hole / slot that is automatically granted.

③ Diameter : It displays the diameter of the selected hole / slot (0.8 ~ 5.0 mm)

④ Depth : It displays the depth of the selected hole / slot. (0.0 ~ 6.0 mm)  
Depth 0.0 mm indicates the hole/slot that penetrates a lens.

⑤ Angle mode : You can choose the angle mode.



Auto (front): drill with the angle of perpendicular to the lens front



Auto (rear): drill with the angle of perpendicular to the lens rear




Manual: drill with the angle of user-specified

⑥ Drilling angle : If you choose 'manual', you can set the drilling angle. (0° ~ 30°)

### NOTE

- Drilling angle

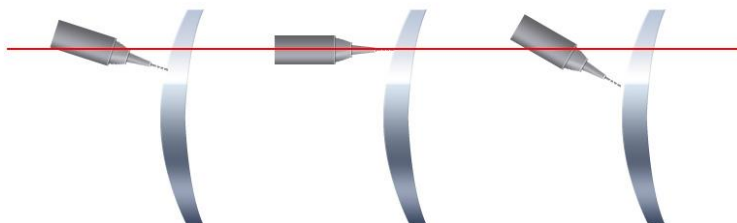


[Auto-front (perp. to the lens front)]

[Manual 0°]







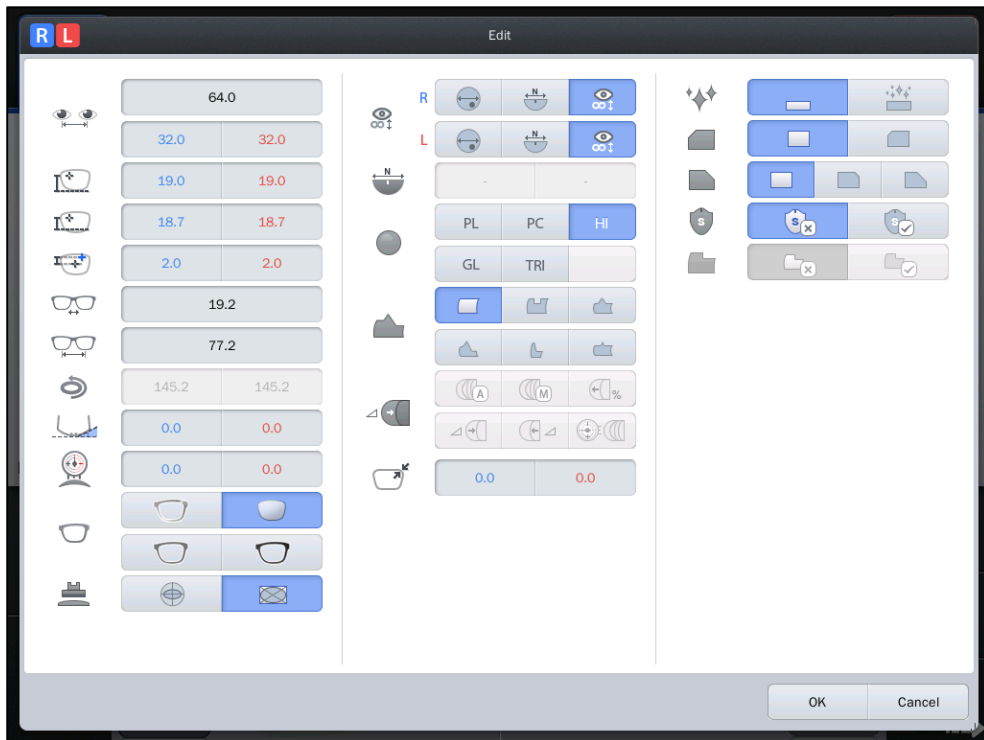



[Manual 30°]

### 5.7. Job Editor




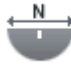
Job editor provides the whole edging options at once, so it is useful for the skillful users.



#### 5.7.1. Layout Option

	PD (Binocular PD, R/L Monocular PD)		Box Height
	$\Delta Y$		Mix Height
	Bridge size (DBL)		FPD
	Frame curve		Circumference
	Frame Tilting Angle		Demo Lens Curve










---

	Frame Type		Blocking mode
	Lens type		Near vision area diameter

---

## 5.7.2. Edging option

---


	Edging type		Lens material
	Bevel/Grooving position		Size
	Polishing		Safety mode
	Front safety beveling		Rear safety beveling
	Step bevel		

---

### NOTE

The job editor options are applied to right/left both side concurrently. But if the options distinguish the two sides, those are applied only to the selected side.

## 5.8. Step Bevel Editor

Step bevel editor provides professional editing function for step bevel edging. The editor screen will be displayed when you select edging type to step bevel or press the editor button (  ) on the bottom of the main screen.



### NOTE

- The Step Bevel Editor is available when the edger type setting is HPE-8000X in the preference menu.
- Preference >> General >> Edger Type : HPE-8000X (Refer to 'Chapter 6.1 Preference')

## 5.8.1. Step Bevel Processing Mode

You can select the step bevel processing mode by option buttons.




Partial Step Bevel : Process user selected region (For half frame goggle)





Full Step Bevel : Process whole region (For full frame goggle)


## 5.8.2. Partial Step Bevel Editing Area

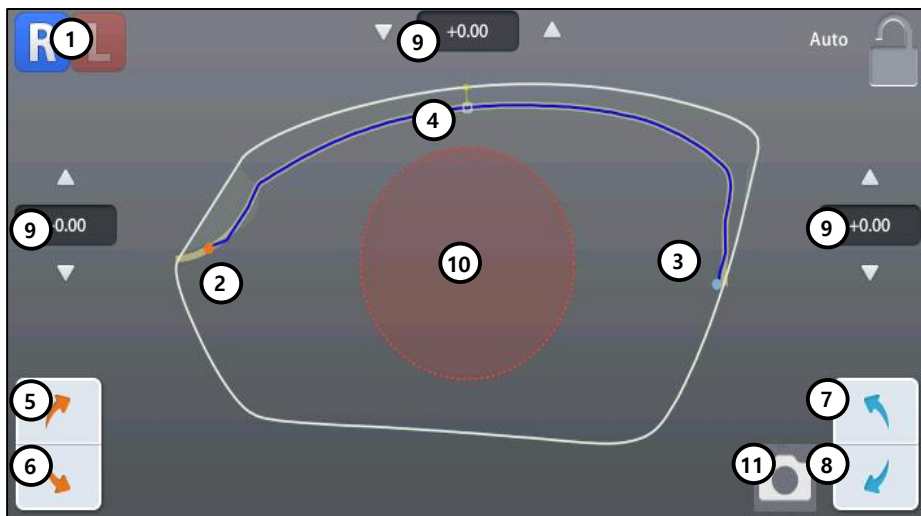


- ① Side selection
- ② The start point of step bevel region (Orange circle)
- ③ The end point of step bevel region (Blue circle)
- ④ Current editing position (White rectangle)
- ⑤ Move start point clockwise
- ⑥ Move start point counterclockwise
- ⑦ Move end point counterclockwise
- ⑧ Move end point clockwise
- ⑨ Step bevel depth input
- ⑩ Edging impossible area
- ⑪ Display Camera Image (On/Off)

**NOTE**

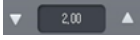- In case of partial step bevel mode, you can enter depth value into three (top/left/right) reference direction. (unit: mm)
- Use the step button(  ) to fine-tune the depth value. (1 step = 0.05mm)
- The fine-tune value of the data obtained by scanning is set to 0.0. It will be edited to a relative amount to original.
- Use camera image button(  ) to display the video of current demo lens.

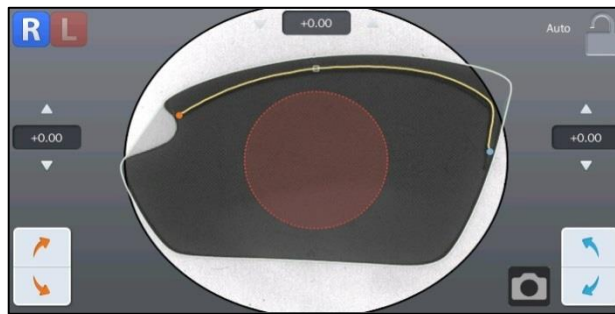


- Display color means as follows:

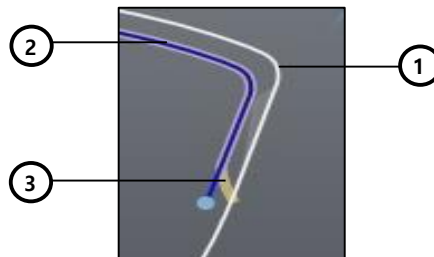


- ① White – Edging shape by main wheel
- ② Blue – Step bevel region selected by user
- ③ Yellow – Final result of step bevel edging

## NOTE

- Touch the lens shape area, then the editing position moves to the touched area.
- To see the shape in actual size, press the upper right icon.

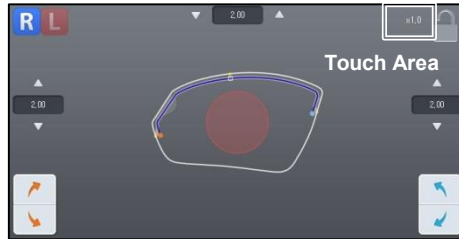


[Actual measurement]

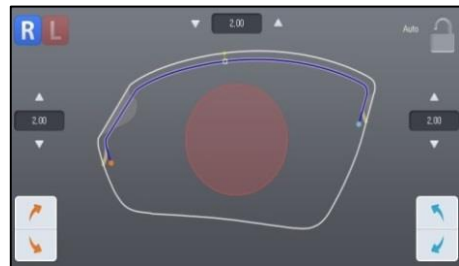


[Auto – Shape resized to fit to the screen]


- Angle guideline pops up when you touch the center of the shape. (Guideline spacing - 10 °)

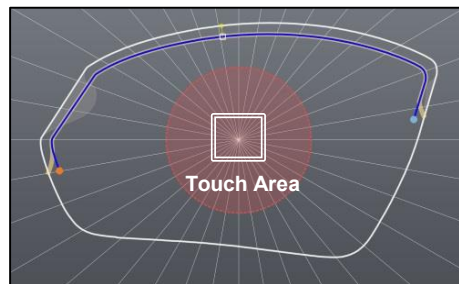

- Touch screen lock button (on/off)

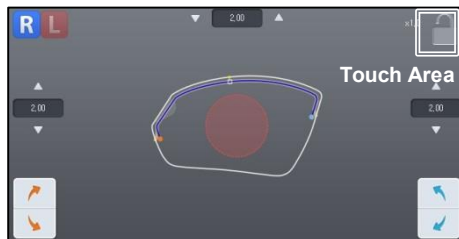

### 5.8.3. Partial Step Bevel Option

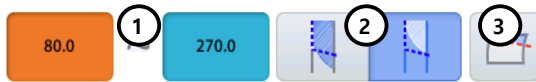

- ① Angle of the start/end point
- ② Step bevel finishing mode (Auto/Safety)
- ③ Inclined cut mode on/off

**NOTE**


- Start point and end point are represented by the angle. And the angle starts with the 6 o'clock position from 0 degree in a clockwise direction.
- Step bevel finishing mode




[Auto mode]



[Safety mode]

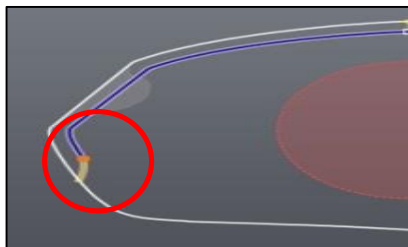


[Auto mode edging result]

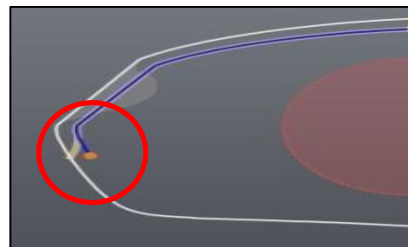

[Safety mode edging result]




- In case of auto mode, both end points of lens will be cut for fitting to frame without any additional user finishing work.
- In case of safety mode, both end points of lens will be cut as minimum amount. Additional finishing work by user is necessary.
- If the inclined cut mode turned on, flat surface of lens will be processed with 12 degree of slope.

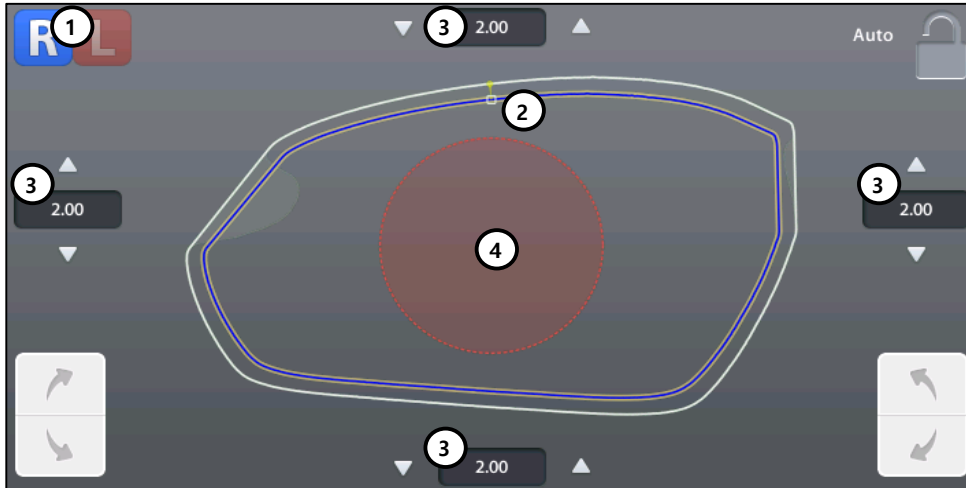
## 5.8.4. Full Step Bevel Editing Area




- ① Side selection
- ② Current editing position (White rectangle)
- ③ Step bevel depth input
- ④ Edging impossible area

### NOTE

- When creating the step bevel data, there are two types ( ) of initial step bevel line for the scan & cut job data.
- After selecting step bevel option,  Button will be displayed. And the step bevel line can follow wheel processing shape or final shape.

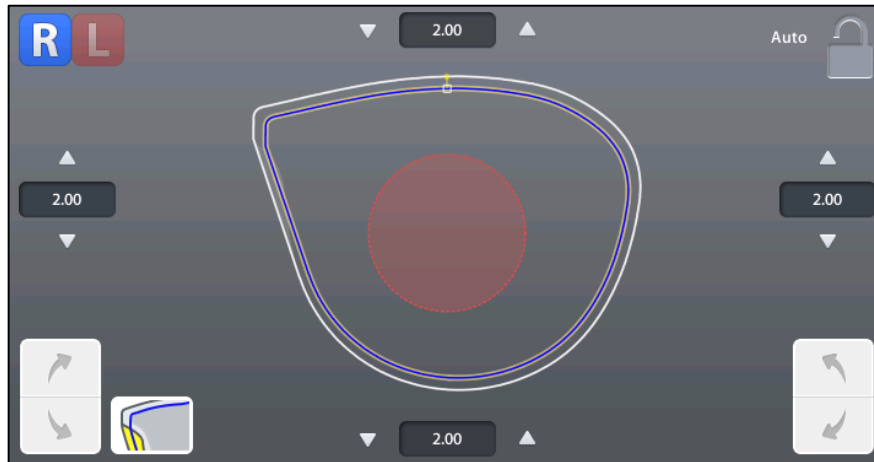

[Following wheel processing shape]

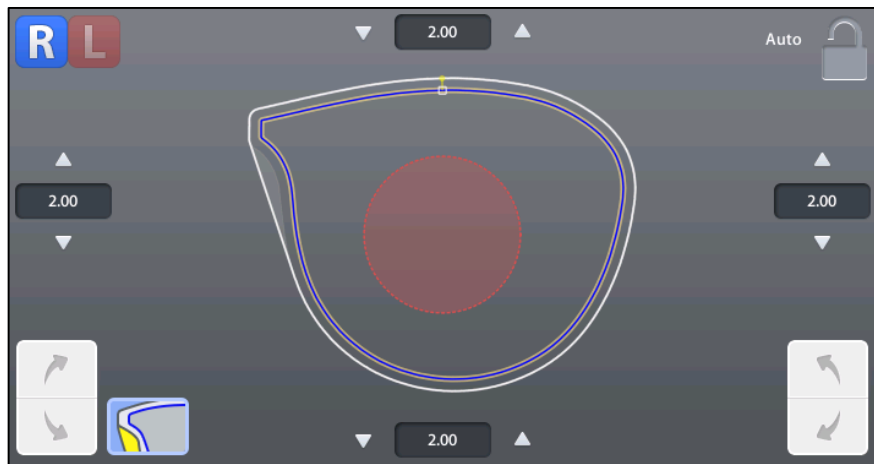

[Following final shape]

## 5.8.5. Full Step Bevel Option




- ① Step bevel depth batch input
- ② Main wheel edging type (Flat / Bevel)
- ③ Inclined cut mode on/off

### NOTE

- In case of full step bevel mode, you can enter depth value into four (top/left/right/bottom) reference direction. (unit: mm)
- With the batch input button you can enter all reference depth values at a time.

## 5.8.6. Fine Tuning


Fine tuning function is used for modifying step bevel shape more delicately.

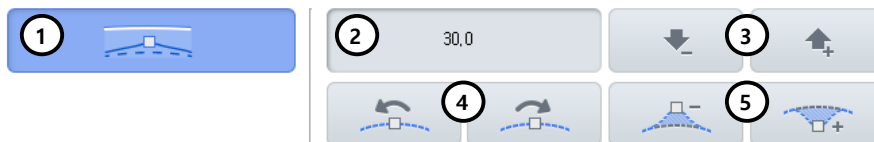




- ① Fine tuning function on/off
- ② Editing scope
- ③ Increase/decrease editing scope
- ④ Move editing position (Counterclockwise/Clockwise)
- ⑤ Shape modification (Convex/Concave)

**NOTE**

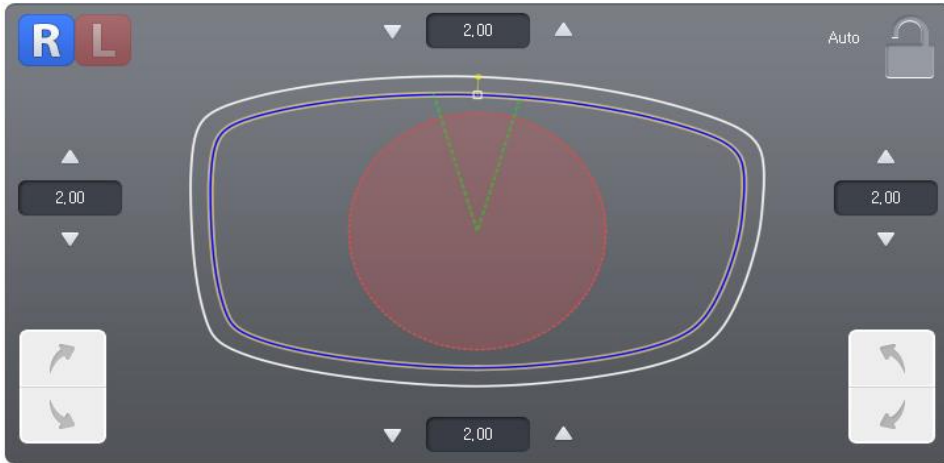
- Editing area with fine tuning function activated



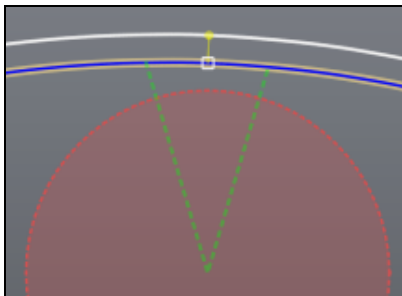

- Editing scope value is represented by the angle and the scope can be changed by increase/decrease buttons (   ).

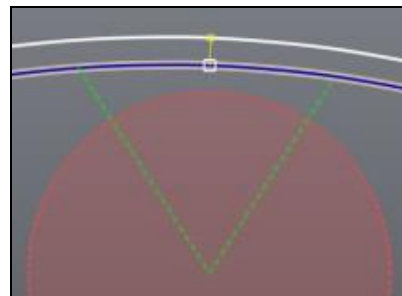


[ 30° scope]




[60° scope]

- Move editing position to target and confirm scope. Then modify step bevel shape by the modification(   ) buttons.


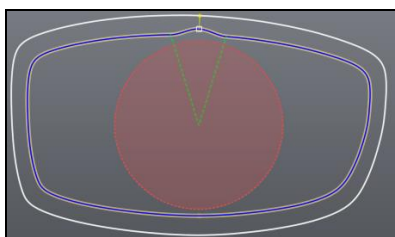


[Convex editing result]

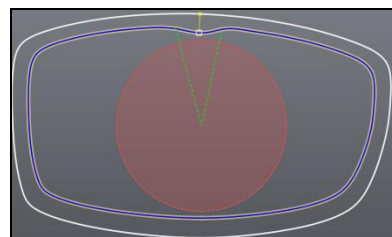



[Concave editing result]

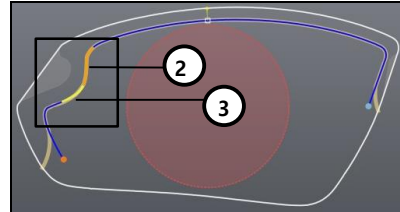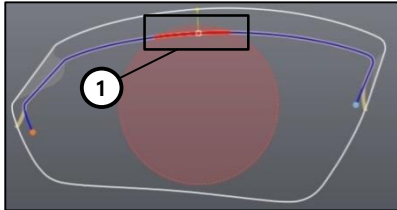
- The color of the step bevel shape can be changed according to the editing result and the meaning of the color is as follows:




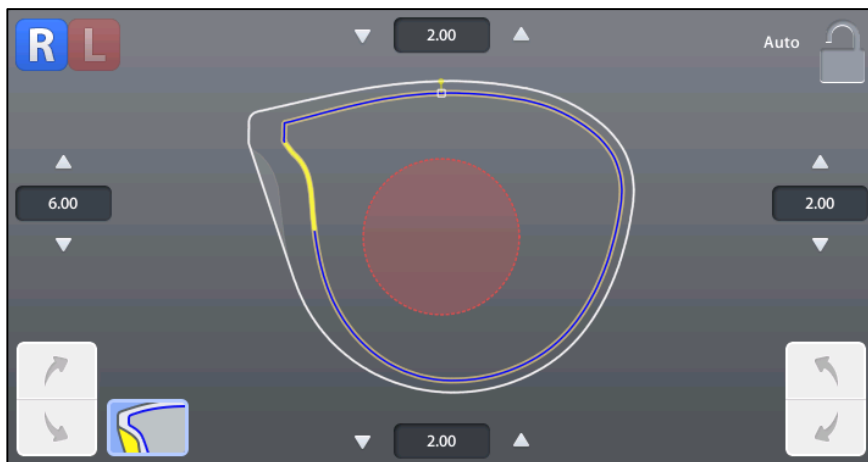
- ① Red : Step bevel shape invades edging impossible area or exceeds depth limit. In this situation, edging is impossible.
- ② Orange : Because of wheel interference the step bevel shape is modified properly. Edging is possible.
- ③ Yellow : This area will be cut during retouch process. (This occurs usually when editing scan & cut job)

**NOTE**


- When there is red line in the editor,  button will be shown in the right bottom and you can edit manually using the edit button or modify the red line automatically by touching the  button.

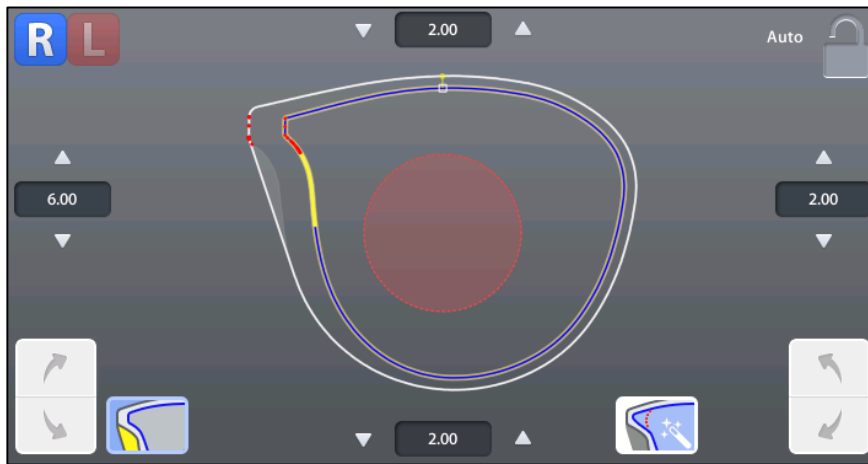

- After touching  button, the red line and the button will be disappeared




## 5.8.7. Preview

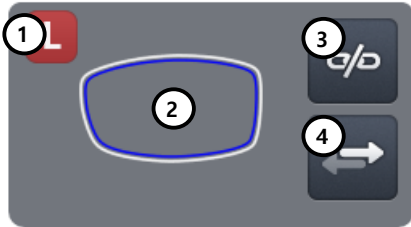


- ① Opposite side display
- ② Edging shape of opposite side
- ③ R/L sync on/off
- ④ Copy to opposite side

### NOTE

- Preview screen always shows opposite side data.
- When R/L sync is turned on, editing result will be applied to both side.
- Copy button (  ) is available when R/L sync option is turned off. Shape of the current side is copied to the other side by pressing this button.
- R/L sync button will be disabled after edging process is done.


### 5.8.8. Current Position Information

It displays the information of current position. You can input additional information for step bevel shape.

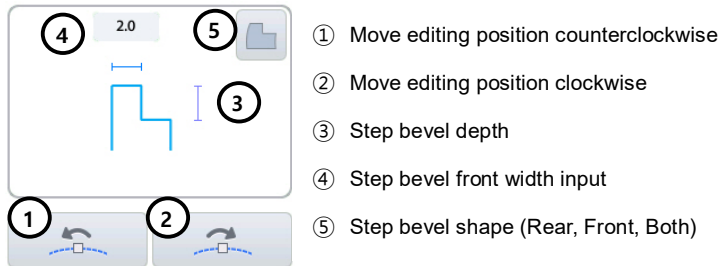


**NOTE**

- Current position information shows lens processing position with the assumption that the lens has sufficient thickness. The actual processing results may vary depending on the lens thickness.
- If the edging type is flat in the full step bevel mode, you can set additional front height.

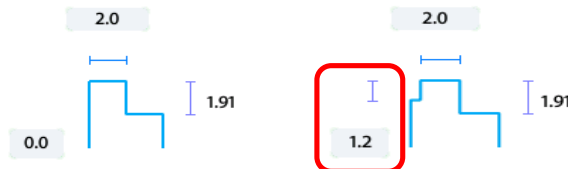

- If the edging type is bevel in the full step bevel mode, you can set additional front height and bevel position. (The bevel position can be set within front width range)

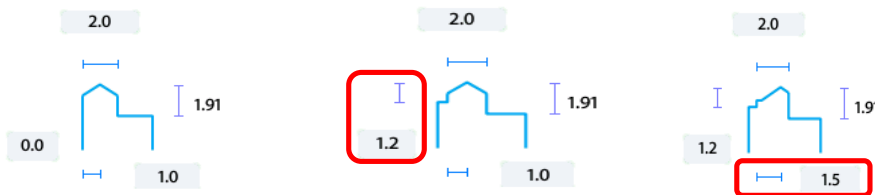

- If you set the front height value, you can adjust bevel position in the manual position modification screen displayed during the edging process.
- In case of partial step bevel mode, main wheel edging type is always flat.
- All the input values of the preview area are applied to the entire region rather than the current position.
- The step bevel depth value shown in the preview area is calculated perpendicular to the

edging surface.

- Shape of step bevel can be selected according to the type of the frame and the lens to be processed.



Rear step bevel



Front step bevel



T-bevel (Both)

- In case of 'Front step bevel' and 'T-Bevel (Both)', you can adjust bevel position in the manual position modification screen displayed during the edging process.

### 5.8.9. Safety beveling & Polishing option

- ① ① Front safety beveling
- ② ② Rear safety beveling
- ③ ③ Polishing

### NOTE

- Safety beveling & polishing may not be available because of the step bevel settings so these options are must be set in the step bevel editor instead of the main screen.

		Front Safety Beveling	Rear Safety Beveling	Polishing
Partial Step Bevel		O	O	O
Full Step Bevel	Flat	O	X	O
	Bevel	O	X	X

- Safety beveling & polishing will be applied on the edging area which processed by main wheel.


### 5.8.10. Reset & Apply Button



- ① Reset Button – Cancel the changes and return to initial state
- ② Apply button – Apply changes and close the editor

#### **NOTE**

---


- When you apply the changes if the result shape is not possible to edging or expected to be modified during the processing, a warning message will be displayed.
- If you want to cancel editing, press the exit button(  ) on the right top of the screen.

#### **CAUTION**

---

The maximum depth of step bevel is 6mm and when the curve of a lens is larger than 6 curve, the maximum depth is 2.5mm.

### 5.9. Bevel Height Editor


Bevel height editor provides professional editing function for asymmetric/semi-u bevel. The editor screen will be displayed when you select edging type to asymmetric/semi-u or press the editor button (  ) on the bottom of the main screen.



 **NOTE**

- The Bevel Height Editor is available when the edger type setting is HPE-8000X in the preference menu.
- Preference >> General >> Edger Type : HPE-8000X (Refer to 'Chapter 6.1 Preference')


### 5.9.1. Editing Area

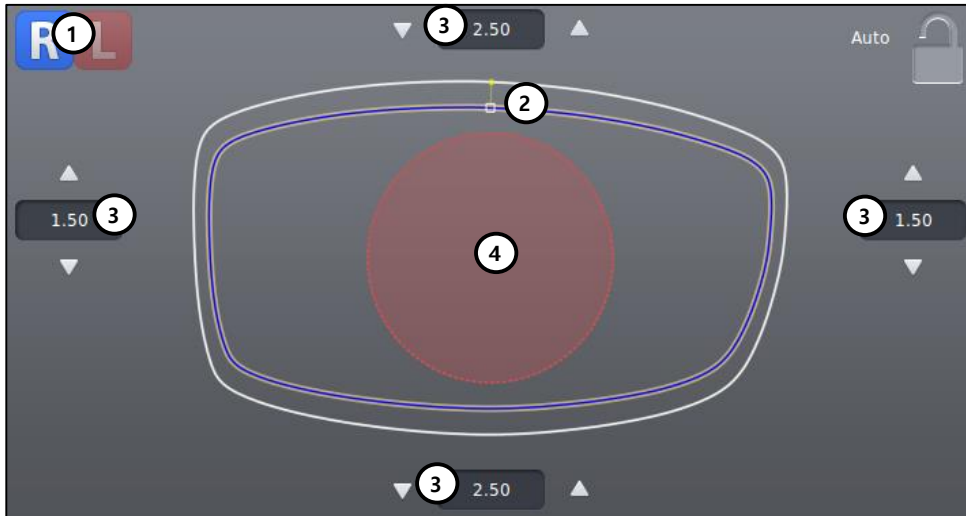

- ① Side selection
- ② Current editing position (White rectangle)
- ③ Bevel rear height input
- ④ Edging impossible area

**NOTE**


- You can enter bevel rear height value into four (top/left/right/bottom) reference direction. (unit: mm)
- Use the step button(  ) to fine-tune the depth value. ( 1 step = 0.05mm)
- Display color means as follows:

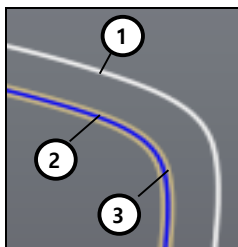


- ① White – Edging shape
- ② Blue – Bevel rear height position
- ③ Yellow – Final result of bevel rear height

## NOTE

- Touch the lens shape area, then the editing position moves to the touched area.
- To see the shape in actual size, press the upper right icon.

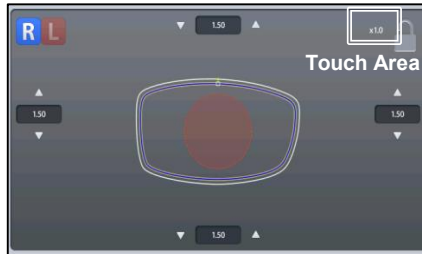


[Actual measurement]

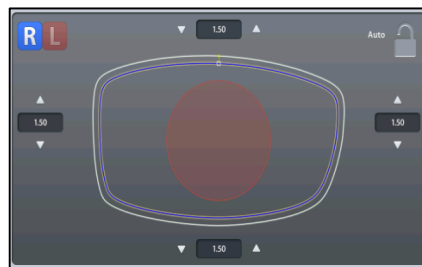


[Auto – Shape resized to fit to the screen]


- Angle guideline pops up when you touch the center of the shape. (Guideline spacing - 10 °)

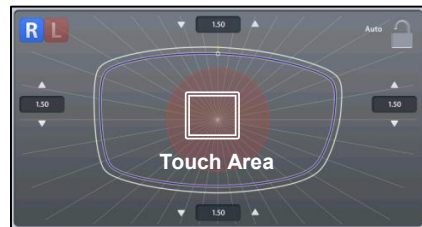

- Touch screen lock button (on/off)

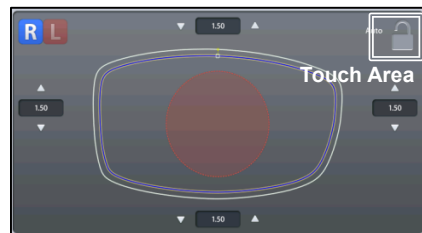


### 5.9.2. Preview

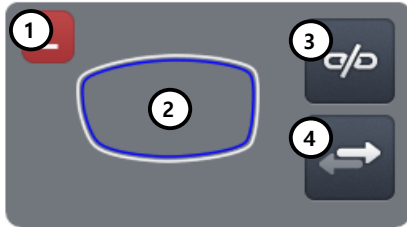

- ① Opposite side display
- ② Edging shape of opposite side
- ③ R/L sync on/off
- ④ Copy to opposite side


---

**NOTE**

- Preview screen always shows opposite side data.
- When R/L sync is turned on, editing result will be applied to both side.
- Copy button (  ) is available when R/L sync option is turned off. Shape of the current side is copied to the other side by pressing this button.
- R/L sync button will be disabled after edging process is done.

### 5.9.3. Current Position Information

It displays the information of current position. You can input additional information for bevel shape.

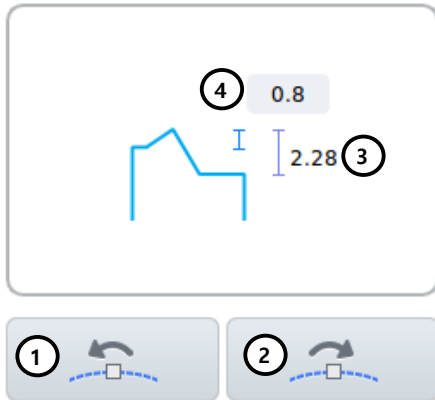

- ① Move editing position counterclockwise
- ② Move editing position clockwise
- ③ Bevel rear height
- ④ Asymmetric bevel front height input (Semi-U bevel front width input)

**NOTE**

- Current position information shows lens processing position with the assumption that the lens has sufficient thickness. The actual processing results may vary depending on the lens thickness.
- In case of semi-u bevel, you can set front width.



- Front input value of the preview area is applied to the entire region rather than the current position.

### 5.9.4. Edging Option



- ① Editing mode
- ② Bevel rear height batch input
- ③ Front safety beveling

#### NOTE


- Currently the editor supports only full region editing mode.
- With the batch input button you can enter all reference height values at a time.

### 5.9.5. Reset & Apply Button




- ① Reset Button – Cancel the changes and return to initial state
- ② Apply button – Apply changes and close the editor

#### NOTE


If you want to cancel editing, press the exit button(  ) on the right top of the screen.

## 6

### Configuration

Press  button on the bottom left of the Main screen.

#### ■ MENU Screen

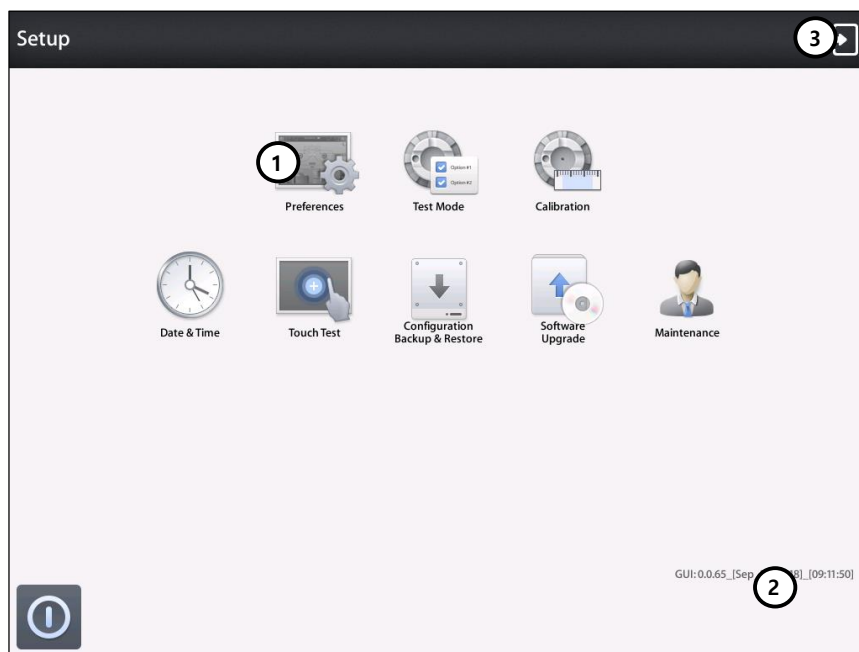

- ① Preferences
- ② Software Version
- ③ Exit the menu screen

#### NOTE


- Refer to 'Chapter 7. System Maintenance' for the other functions.
- Please be careful not to suddenly turn off the power while using the device.
- Unexpected shutdowns can cause internal configuration data corruption.
- Please press the shutdown button on the menu screen before turning off the power switch.

## 6.1. Preference

User Interface can be customized to the user's preference.

### General

Default settings for User Interface option.

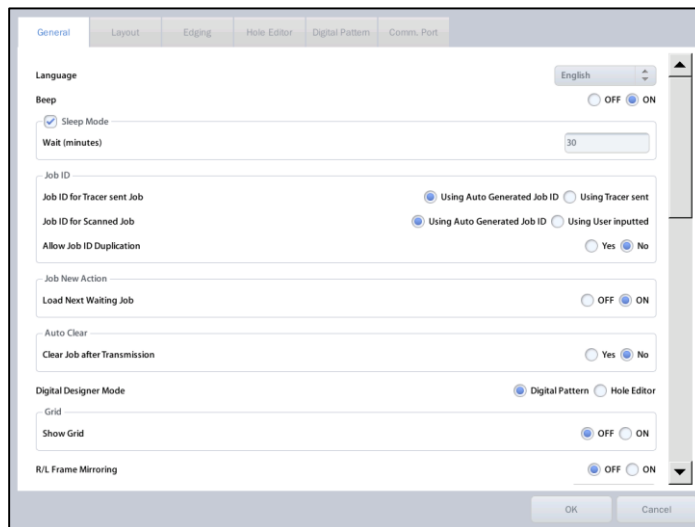

- Language – Sets default language.
- Beep – Turns on or off the beep sound.
- Sleep Mode – Inputs sleep mode waiting time. (minutes)
- Job ID for Tracer sent Job – Set ID for the tracer sent Job.
- Job ID for Scanned Job - Set ID for the digital scanned job.
- Allow Job ID Duplication - Set whether to allow job number duplication.
- Job New Action - Set the 'New' button behavior.
- Auto Clear - Clear main screen after transmission.
- Digital Designer Mode – Sets digital designer startup screen.
- Grid – Shows or hides gridlines of main screen.
- R/L Frame Mirroring - Mirroring the shape. (Not the size)
- Scan Area Margin (Digital Scan) - Set digital scan area margin.
- Select Edger Type – Select wheel type of the Edger connected to the blocker.
- Scan Option - Set digital scan option.
- Lens Adaptor Type - Set lens adaptor type.
- Export Options – Set options for JOB export to SD card.
- Message Box Font Size - Set message dialog box font size.
- Config DB History - Displays configuration data backup/restore data.
- Manufacture ID – Displays machine ID.



## NOTE

- Setting Screen


The screenshot shows a software interface with several sections and controls:

- 1**: Tab Menu (General, Edging, Hole Editor, Digital Pattern, Comm. Port)
- 2**: List Box (Single Vision)
- 3**: Radio Button (Boxing Center, Optical Center)
- 4**: Input Box (Delta Y, 2.0)
- 5**: OK Button
- 6**: Cancel Button

- ① Tab Menu.
- ② List Box – Choose a list from existing options.
- ③ Radio Button – Choose only one of a predefined sets of options.
- ④ Input Box – Input value to be used by the program.
- ⑤ OK Button – Save the changes.
- ⑥ Cancel Button – Cancel the changes.

## Layout


Default settings for Layout options.

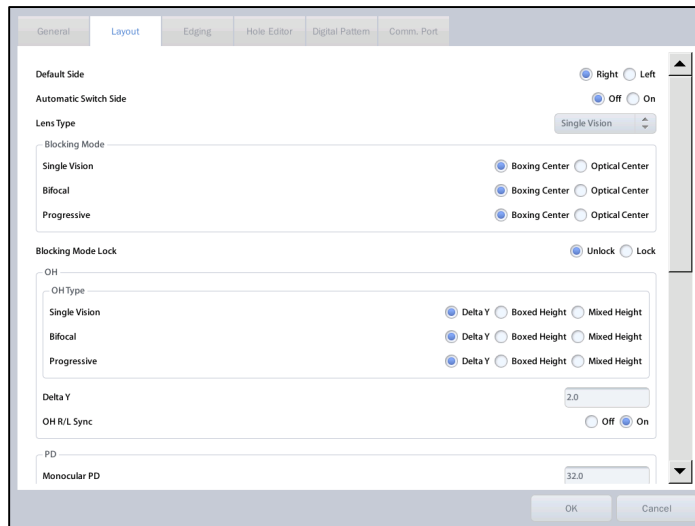

- Default Side – Select the default side for a new job.
- Automatic Switch Side – Set automatic side change after blocking.
- Lens Type – Select the default lens type for a new job.
- Blocking Mode – Select the default blocking mode for a new job for each lens type.
- Blocking Mode Lock - Turn on the blocking mode lock function.
- OH – Set the default input style for the optical height and Delta Y according to the lens type.
- OH R/L Sync : Turn on or off the function of the R/L synchronization.
- PD – Set the default value of PD.
- Bridge Size - Set the default value of bridge size.
- Bifocal Lens – Set the default value for Bifocal Lens. (Vertical and Horizontal Offset and Near vision lens diameter)
- Manual Blocking – Set guide line for manual blocking.
- Mandatory Field Warning – Check if mandatory values are entered or not.
  - PD – Warning message pops up if PD value is not entered.
  - OH – Warning message pops up if OH(Delta Y, Boxed Height or Mixed Height) value is not entered.
  - Bridge Size – Warning message pops up if Bridge size or FPD value is not entered.

## ■ Edging

Default settings for Edging options.

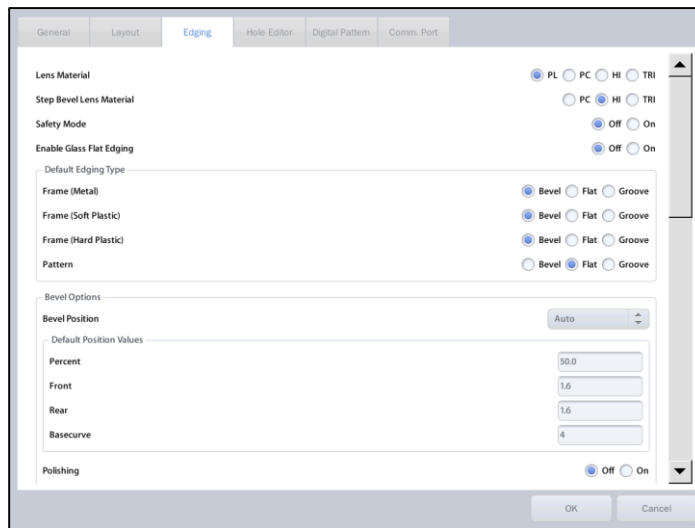


- Lens Material – Set the default lens material.
- (Opt) Step Bevel Lens Material - Set the default lens material for step bevel.
- Safety Mode – On or Off the safety mode.
- Enable Glass Flat Edging – On or Off the flat edging for glass lens.
- Default Edging Type – Set the default edging type for frame and pattern.
- Bevel Options – Set default values for bevel edging.
- Rimless Options – Set default values for rimless edging.
- (Opt) Asymmetric Bevel Default Value - Set default values for Asymmetric bevel.
- Mini Bevel Default Value - Set default values for Mini bevel.
- (Opt) Semi-U Bevel Default Value - Set default values for Semi-U bevel.
- (Opt) Step Bevel Default Value - Set default values for Step bevel editor.



\*(Opt) marked options are displayed with specific edger type

## Hole Editor

Default settings for Hole Editor options.

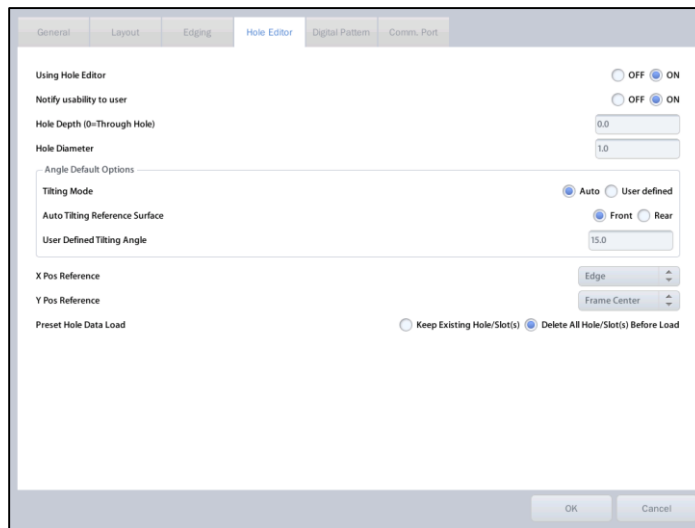


- Use Hole Editor – Hole Editor On/Off.
- Notify usability to user - Displays the hole editor enable message box.
- Hole Depth – Set default hole/slot depth value.
- Hole Diameter – Set default hole/slot diameter value.
- Angle Default Options – Set default tilting angle mode and manual tilting value.
- X Pos Reference – Set default horizontal coordinate reference.
- Y Pos Reference – Set default vertical coordinate reference.
- Preset Hole Data Load – Set default behavior of preset loading.

## ■ Digital Pattern

---


Default settings for Digital Pattern options.

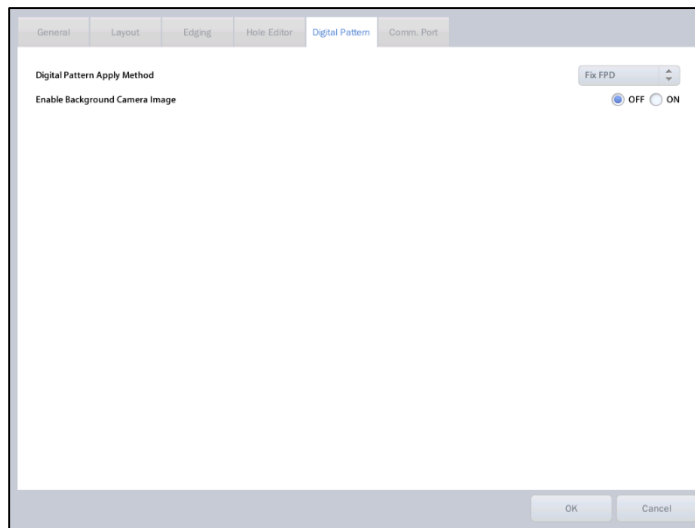

- Digital Pattern Apply Method.
  - Fix FPD – Calculates bridge size value based on FPD.
  - Fix Bridge Size – Calculates FPD value based on bridge size.
- Enable Background Camera Image

## Communication Port

Default settings for communication options.

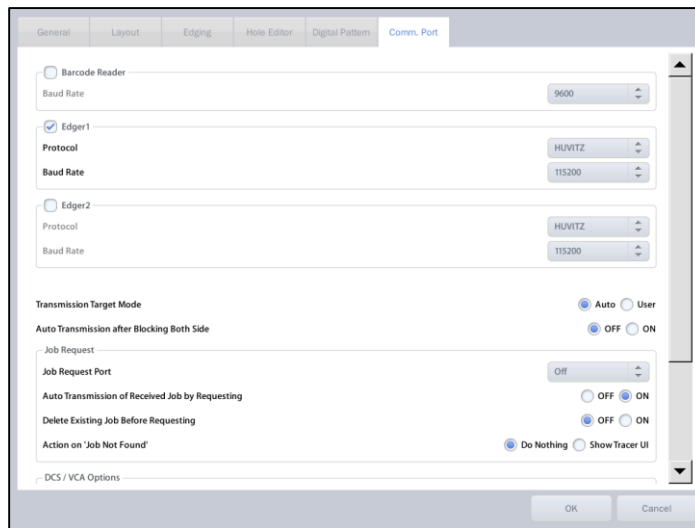

- Barcode Reader - Use Barcode Reader port.
  - Baud Rate - Set communication speed.
- Edger1 – Use Edger1 port.
  - Protocol – Set communication protocol.
  - Baud Rate – Set communication speed.
- Edger2 – Use Edger2 port.
  - Protocol – Set communication protocol.
  - Baud Rate – Set communication speed.
- Transmission Target Mode – Select the mode if the blocker uses two edger ports.
  - Auto mode - Transmit job to both edgers.
  - User mode - Transmit job to the selected edger.
- Auto Transmission after Blocking Both Side – On or Off the auto transmission after blocking both side.
- Job Request - Set communication port for job downloading and default behavior.
- DCS / VCA options – Set communication options for LMS.
  - Machine ID – Set machine ID.
  - The Number of Radii – Set the number of radii.
  - Radius Mode – Set angle type.
  - Format – Set data format.
  - Upload Job – Set Enable/Disable Upload Job function to Host PC.


# 7

---

## System Maintenance

### 7.1. Brightness Test & Calibration

You can diagnose the current brightness and calibrate it.

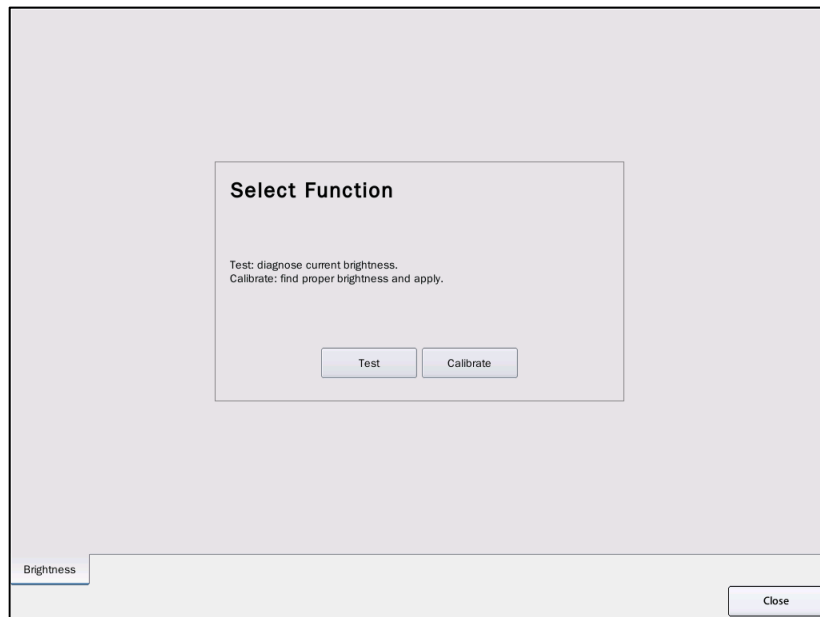


## 7.2. Save Image

When the digital scan quality is low, you can save the image file to SD card and request improvement.

### ■ Save Digital Scan image


During digital scan, you can save digital scan image if you want. (SD card must be inserted in slot)

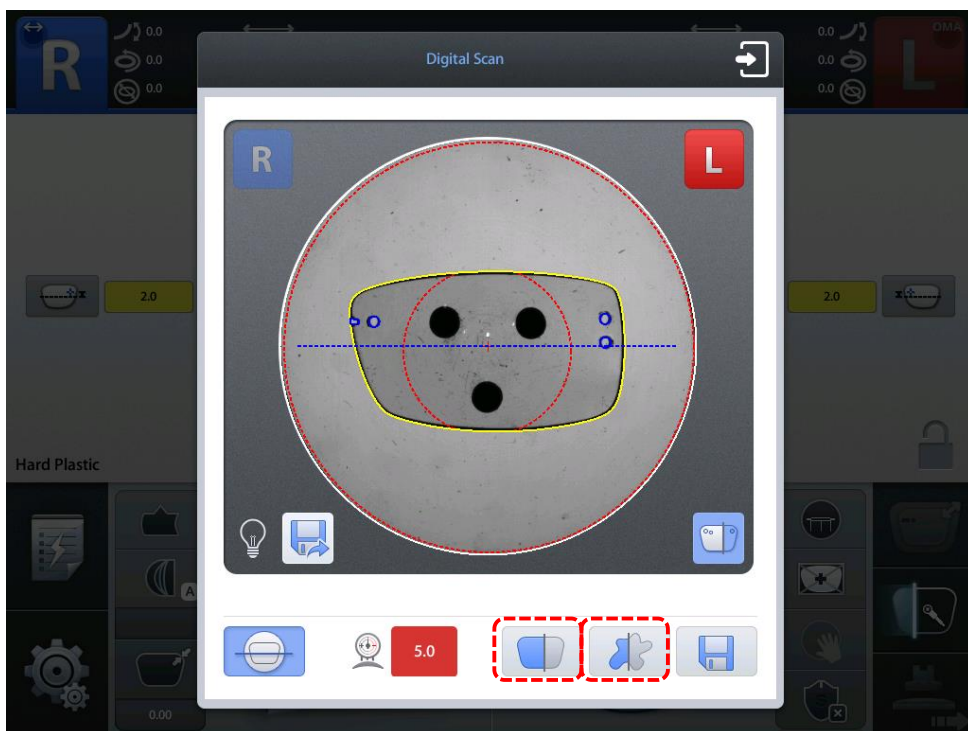

- ① Please press the scan button on digital scan window for a long time, then “Holding Down the Button” message will be displayed.
- ② At this moment, release the button, then the still image is saved in SD card

## 7.3. Test Mode


You can check the sensor info of the Blocker and test the motor of the reflector.

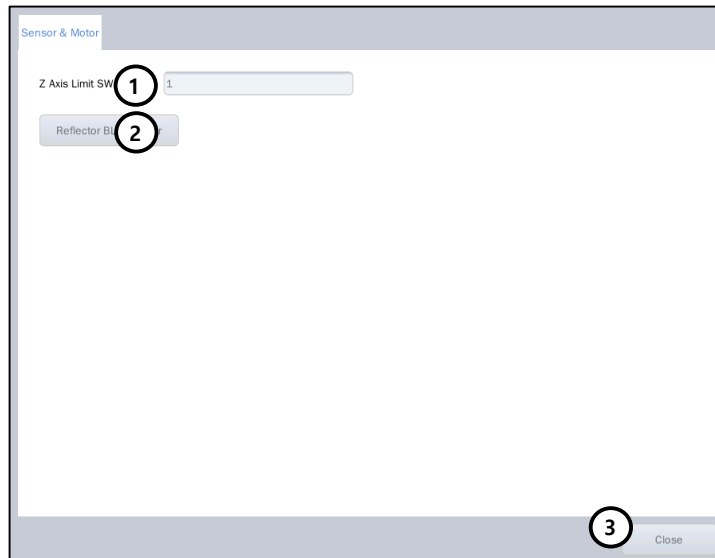

- ① Z Axis Limit Switch Value
- ② Reflector BLDC Motor On/Off
- ③ Exit Test Mode

## 7.4. Date & Time


You can change the system time. Please set the date and time first after you install this machine.

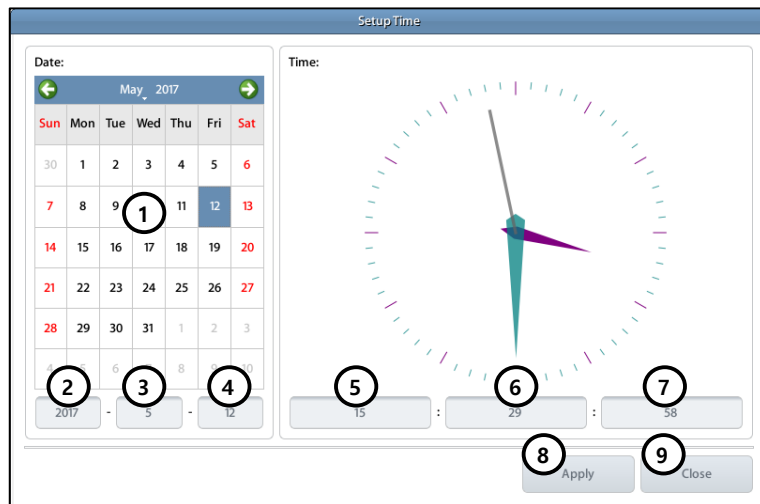

- ① Date selection calendar
- ② Year
- ③ Month
- ④ Day
- ⑤ Hour
- ⑥ Minute
- ⑦ Second
- ⑧ Apply modified data & time
- ⑨ Exit the menu without saving the changes

## 7.5. Touch Test

You can check the normal operation of the touch screen

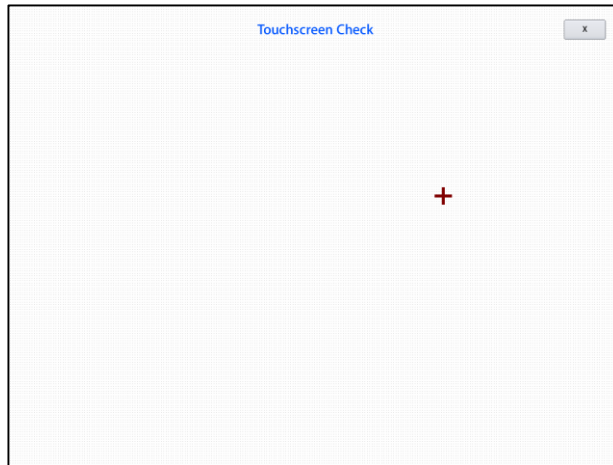


### NOTE

- You can use the emergency touch screen test function if the touch test is not possible
  - ① Turn off the device and turn on the power switch while pressing the touch screen.
  - ② If you keep pressing the screen while the initialization screen is displayed, the emergency touch test message will be displayed on the screen.
  - ③ If you keep pressing the screen as shown on the popup message, the emergency touch screen test function will start.

## 7.6. Configuration Management

You can backup or restore system configuration data.

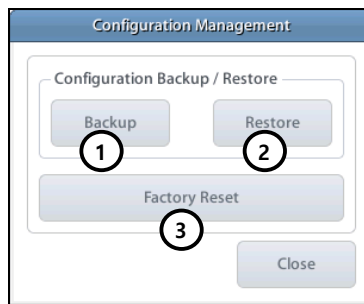

- ① Backup to SD Card – Backup configuration data to SD card.
- ② Restore from SD Card – Restore configuration data from SD card
- ③ Factory Reset – Restore to factory default setting.

### NOTE

- Configuration Management function is available when SD card inserted.
- You can backup the configuration data as many times as you need, because you can specify file name.

## 7.7. SW Upgrade

You can update to the latest software.



① GUI – Upgrade GUI software

### NOTE

- It is highly recommended to use latest software.
- If you have multiple versions of SW in the SD card, you can select the version by pressing the version button.
- How to upgrade software
  - ① Copy upgrade file to a SD card and insert the SD card into the system.
  - ② Check the SW version displayed on the upgrade dialog box
  - ③ Press the upgrade button to start upgrade.
  - ④ The upgrade process will take some time. (10 ~ 20 seconds)
  - ⑤ After upgrading process finishes, restart the system.
  - ⑥ If the system rebooted, new software will be launched.

## 7.8. Maintenance Code


Maintenance code is used for the engineer to diagnose the system.



## 7.9. Maintenance Tips

### 7.9.1. Fuse Replacement

Be sure to check the Fuse Specification is 250V 3.15A. Fuse box is located below the power plug at the rear side of the blocker.

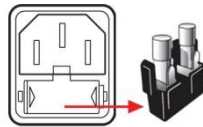

### WARNING

Be sure to use the appropriate Fuse. Otherwise it may damage the product or cause fire.

### 7.9.2. Regular maintenance

It is recommended to keep the regular maintenance cycle in order to maintain the Edging Performance. Be sure to carry out the regular maintenance by the service technician of HUVITZ or the technician authorized by HUVITZ.

### 7.9.3. Cleaning

- ① Sweep the outer surface and LCD panel with soft fabric material.
- ② Remove dust on the lens supporter and reflector with soft fabric material.
- ③ When you do not use this device, wrap it with dust cover.



### CAUTION

- Before cleaning your product, disconnect the power cord.
- Since the exterior of the product is easily scratched, be sure to use soft cleaning cloth.
- When cleaning the product, avoid spraying cleansing agent directly onto the product. This may result in the exterior being deformed or the print being removed.
- Since using a strong chemicals, may result in the discoloration or cracking of the product exterior, be sure to use the clean water only.

# 8

## Troubleshooting Guide

If problems occur, please review the following list and take remedial action as needed. If you cannot solve the problem after checking the entire list, please contact your authorized distributor or HUVITZ.

Problem	Cause	Solution
<p>When blocking, the lens adaptor is not attached to the lens</p>	<p>① Adhesive strength has been weakened because the leap tape is reused.</p> <p>② Ball plunger is too tighten.</p> <p>③ When blocking a high-curve lens to box center, the lens adaptor is partially attached to the lens.</p>	<p>① Do not re-use the leap tape. (If you re-use it, dry off the leap tape sufficiently by using heater)</p> <p>② Release the ball plunger a little bit so that the lens adaptor falls out smoothly. (If the ball plunger is loosened too much, the center and axis can be shifted. So it is necessary to check the center and axis are normal after unfastening.)</p> <p>③ When blocking the high-curve lens, use small lens adaptor instead of long lens adaptor.</p>
<p>When blocking, the holder does not hold the lens adaptor securely and the adaptor falls off well.</p>	<p>Ball plunger is too loosen.</p>	<p>Tighten the ball plunger to hold the lens adaptor securely. (Be sure to check the center and axis are normal after fastening.)</p>

# 9

---

## Specifications and Accessories


### 9.1. Standard Accessories

User's Manual .....	1
Lens Supporter .....	1
Lens Supporter for Frame Change and Digital Scanning .....	1
Touch Pen .....	1

### 9.2. Specifications

Imaging Camera	CMOS B/W 1.3M
Blocking Tolerance	-0.5 ~ +0.5mm
Axis Tolerance	$\pm 1^\circ$
Blocking Method	Manual
Display	9.7inch Color TFT LCD (1024x768) with Touch Screen, Tilting
Dimensions	236(W) x 268(D) x 422(H)
Weight	8Kg
Power Supply	AC 100~240V 50/60Hz
Power Consumption	20W

9.3. Drawings of System

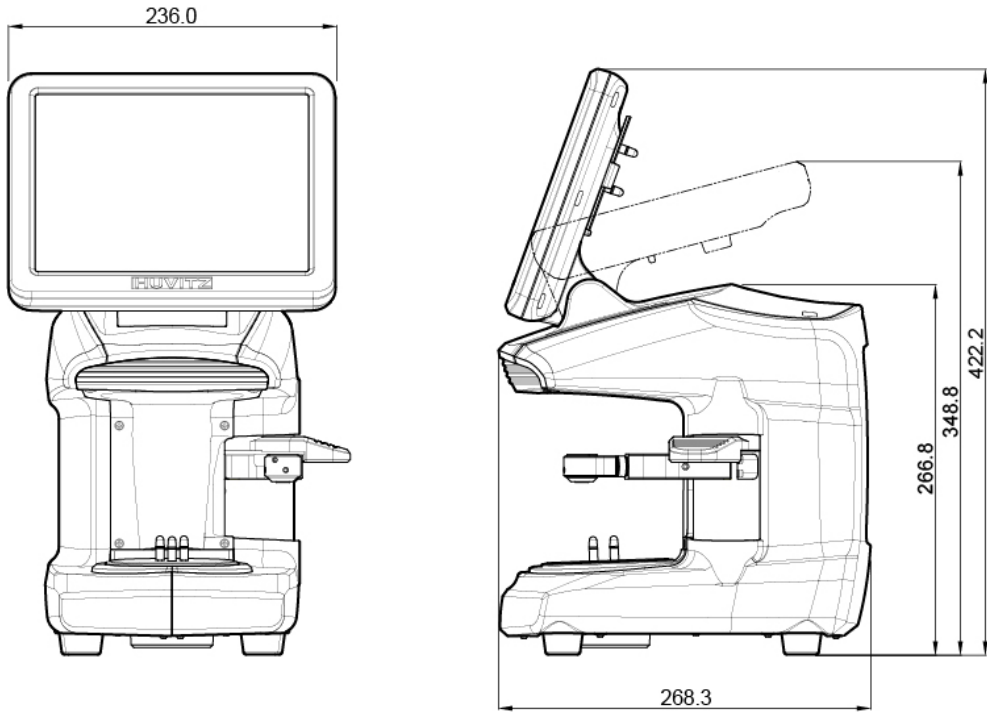

# 10

---

## SERVICE INFORMATION

If the instrument appears malfunctioning, it is highly recommended to check the instrument according to the troubleshooting procedure in chapter 8 of this manual. If any problem persists or the instrument is damaged or malfunctioning, please follow the steps below.

- Please contact the local distributor in your province or country at first.
- Before calling to the local distributor, please be sure to check Model and Serial Numbers.
- The serial number is unique to this unit and printed on the left side of the unit. It is recommended to fill up the following table as soon as you purchase our product.
- Please keep this manual as a permanent record of your purchase and keep your purchase receipt as your proof of purchase.

**Date of Purchase:** \_\_\_\_\_

**Dealer's Name:** \_\_\_\_\_

**Dealer Address:** \_\_\_\_\_

**Dealer Phone No. :** \_\_\_\_\_

**Model No.:** \_\_\_\_\_

**Serial No.:** \_\_\_\_\_

- If you can't contact with your local distributor, you can directly get in touch with the service department of the HUVITZ using the phone number and the address written in the below table.

### ■ How to Contact HUVITZ Co., Ltd

---

**HUVITZ Co., Ltd.**  
38, Burim-ro 170beon-gil, Dongan-gu,  
Anyang-si, Gyeonggi-do, 14055,  
Republic of Korea

Tel: +82-31-428-9100  
Fax: +82-31-477-9022(FA)  
e-mail: [svc@huvitz.com](mailto:svc@huvitz.com)  
<http://www.huvitz.com>